



ProShield™

PROTECTOR & CONTAMINANT SHIELD
FOR ENDOSCOPES



AN
LEAN
CL



Improving Care. Improving Business.

ProShield™

PROTECTOR & CONTAMINANT SHIELD FOR ENDOSCOPES

The ProShield is a simple, single-use container that improves handling protocols helping to reduce the amount of scope transport damage. This endoscope sheath aids in reprocessing and storage of flexible endoscopes and TEE Probes. Additionally, the hazards of delayed reprocessing are reduced, patient and staff safety is improved, cross contamination liability is addressed, and repair costs are managed.

FEATURES & BENEFITS

- Helps eliminate costly endoscope transport damage
- Funnel style opening for quick and effortless insertion of scope, perfect for secured transport of scope
- Individually sealed to keep inside of ProShield clean, preventing micro-organism carrying dust from settling on or in tube surfaces
- Peel-pack seal for quick removal and use during stat rushed conditions
- Sealed bottom contains contaminants and allows use of a rinsing solution to quickly remove materials before they harden and dry
- Enhanced and improved patient and staff safety
- Effortless scope storage allows re-introduction
- Enables immediate enzyme soap soaking of scope at patient bedside
- Helps reduce hospital-acquired infections and liability
- Patent pending
- Latex free




ProShield

Product No.	Description	Quantity
9600-030A-10D	ProShield, Sealed Bottom, 30"	10/Cs
9600-030A-50D	ProShield, Sealed Bottom, 30"	50/Cs
9600-048A-10D	ProShield, Sealed Bottom TEE, 48"	10/Cs
9600-048A-50D	ProShield, Sealed Bottom TEE, 48"	50/Cs

ProShield Hardware

Product No.	Description	Quantity
9600-BU10D	ProShield Universal C-Bracket	1/Cs
9600-BP10D	ProShield IV Pole Mount	1/Cs
9600-SD06D	ProShield Table Top Stand	1/Cs
9600-CW03D	ProShield Cabinet	1/Cs

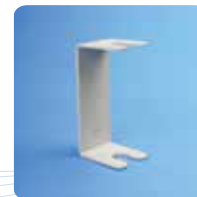

9600-BU10D



9600-BP10D



9600-SD06D



9600-CW03D



Improving Care. Improving Business.

GETTING STARTED WITH THE PROSHIELD



1. Open the top of the peel-pack.



2. Insert flexible shank into ProShield and stretch rubber band over the handle.



3. Clip the ProShield with device into the bracket.



4. Wall storage of clean devices.



5. Cabinet storage of devices in ProShields.



6. Stand for quick storage and easy transport.



7. Remove the "CLEAN" label when procedure starts.



8. Add cleaning solution by lifting handle and pour to pre-soak the scope.

ProShield™

PROTECTOR & CONTAMINANT SHIELD
FOR ENDOSCOPES



ProShield

DeRoyal offers the ProShield to proactively address the FDA's safety communication to reduce cross contamination. The ProShield is a value-laden, single-use endoscope transport system that improves handling protocols and helps reduce the amount of scope transport damage. ProShield further aids in the reprocessing of flexible scopes by allowing the clinician to begin the cleaning process at the patient's bedside, reducing the risk of hospital acquired infections. Proper storage and transport of flexible endoscopes, including TEE Probes, begins and ends with the ProShield.



Improving Care. Improving Business.

DeRoyal Industries, Inc.
200 DeBusk Lane | Powell, TN 37849
888.938.7828 or 865.938.7828
www.deroyal.com

Reprint #0-2318
Rev. 12/2012