

CLEANING & DISINFECTION / INSPECTIONS / WARRANTY

CLEANING & DISINFECTION



WARNING

Before cleaning with any liquid cleaner be sure to unplug the power cord from the outlet.

Use a 10% sodium hypochlorite (bleach) solution or *Recommended Disinfectants* on all surfaces. Clean all sides of each upholstered section. Follow the directions on the disinfectant and wipe off excess.

Recommended Disinfectants

Protex, MadaCide, Accell TB, Virox®

Note: Damage caused by unapproved substances will not be covered under the warranty.

DO NOT use citrus based cleaners or other strong cleaners, such as alcohol, acetone, higher concentrations of bleach or other products that contain high concentrations of these substances.

DO NOT expose the fabric to temperatures below 50°F/10°C or above 104°F/40°C.

DO NOT expose the fabric to direct sunlight, adhesives, liquids, or abrasive materials.

INSPECTIONS & MAINTENANCE

RECOMMENDED REGULAR INSPECTIONS (monthly or local standard)

- Check for damage to the power, hand control or foot control cables.
- Visually inspect components for obvious damage that could cause problems during operation.

RECOMMENDED PERIODIC INSPECTIONS (yearly or local standard)

- Check for damage to the power, hands control or foot control cables and all visible wiring.
- Visually inspect components for obvious damage that could cause problems during operation.
- Check all mechanical functions using the hand control. Repeat using the foot control. Check for abnormal noises.
- Replace any missing or illegible labels.
- Check that all fasteners are present and fastened securely.
- Check table grounding.
- Clean unusual buildup of dirt on the chair and/or parts of the chair not normally cleaned on a regular basis.
- Check for tears or cracks in the upholstery.

MAINTENANCE

- No specific maintenance tasks are required.
- Oakworks medical tables/chairs are designed and built to provide many years of dependable service. Please follow local regulations and laws when disposing of the electrical components used in this chair.
- ***For all Customer Service related problems refer to the Service Manual***

WARRANTY

View complete warranty details at www.oakworks.com