Reusable Overlay Cover Instructions for Use

For additional information about using this cover, refer to the instructions provided with a compatible ROHO Mattress Overlay.

Supplier: This manual must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Incident Reporting: In the event of a serious incident or injury, contact Customer Support.

⚠️ Warning: Indicates that not following the specified instructions may lead to hazardous conditions resulting in serious injury.

Cover Placement

⚠️ Warnings: Periodically, inspect the cover for damage and replace if necessary. • If you are unable to perform any steps, seek assistance. • The mattress overlay surface will accclimate to the surrounding temperature. Take appropriate precautions, especially when the overlay will contact unprotected skin. • DO NOT use the product as a transfer surface.

1. Before the individual transfers to the bed, place the cover around the overlay and the bed mattress, referring to the directional labels to orient the cover.
2. If needed, the following items may be placed on top of the Mattress Overlay and Reusable Overlay Cover: One flat or contoured sheet, arranged loosely • One incontinent pad • One draw sheet, for moving and transferring.

⚠️ Warning: Placing too many linens between the individual and the product will reduce its effectiveness.

Cover Removal: The individual should transfer to a different surface. Pull each corner away from the bed mattress and overlay.

Cover Care

⚠️ Warnings: Clean the product regularly and when it becomes soiled. If the product becomes soiled, or between use by different individuals: clean, disinfect, and check for proper functioning. • Follow all manufacturer bleach container and germicidal disinfectant safety guidelines. • This product does not require sterilization; DO NOT autoclave. • DO NOT dry clean.

Notes: Repeated washing at high temperatures can accelerate deterioration of the cover and is not recommended. • Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer.

Symbols Glossary

REF Model number	FITS Fits	Date of manufacture	Manufacturer	⚠️ Fall risk.	Machine wash hot, normal, at the temperature shown.	Hand wash.	Line dry.
MD Medical Device	WDH Size	<input checked="" type="checkbox"/> Meets flammability		⚠️ Entrapment risk	Machine wash cold, gentle.	Tumble dry low.	CL Cleaning instructions
LOT Batch code	<input checked="" type="checkbox"/> Consult instructions for use.	<input checked="" type="checkbox"/> Authorized representative in the European Community		⚠️ Suffocation risk	Bleach (1 part bleach: 9 parts water).	Do not dry clean.	DI Disinfecting instructions

Limited Warranty Term: 6 months; see also the Limited Warranty Supplement

Product information is changed as needed; current product information is available at permobilus.com. Contact Customer Support for previous versions of product information. • ROHO® is a registered trademark of Permobil. • ROHO, Inc. is a part of Permobil Seating and Positioning.

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