

STATIONARY OXYGEN CONCENTRATOR FILTERS

The LM5CA 5L Stationary Oxygen Concentrator has 3 filters. For optimal performance and longevity of the unit and to preserve the Manufacturer's Warranty, Rhythm Healthcare provides the following guidance for filter cleaning and replacement.



CABINET FILTER

SKU LM209069

The Cabinet filter is located on the left side of the unit. The filter cover needs to be removed with a small screwdriver to access the filters. The Cabinet Filter should be removed, washed with warm water and dish soap every two weeks. It needs to be COMPLETELY dry before being replaced. The filter should be replaced when it is torn or otherwise damaged and in between patients.



AIR INTAKE FILTER (HEPA FILTER)

SKU LM209066

The Air Intake (HEPA) filter is located behind the cabinet filter on the left side of the unit. It should be inspected regularly and replaced annually or when it turns black. The Air Intake (HEPA) filter should also be changed in between patients.



BACTERIAL FILTER

LM209068

The Bacterial Filter is located inside of the LM5CA Concentrator. This filter should be changed with every compressor servicing and in between patients.

