

# HANDHELD SHOWER HEAD

## QUICK START GUIDE

### WHAT'S INCLUDED


- 1x Mounting Bracket
- 1x Hose
- 1x Shower Head
- Thread Tape Roll

### OVERVIEW


We are constantly answering questions and recording helpful videos to make using your Vive Handheld Shower Head as easy as possible. Check out the included link and QR code to help you through the process.

### INSTALLATION INSTRUCTIONS


1. Remove the old shower head and clean the existing threads off the water pipe. Then apply a new layer of thread, 3-4 times around the pipe is enough.
2. Before installing the mounting bracket to the existing water pipe, make sure that the two parts of the mounting bracket are connected tightly.

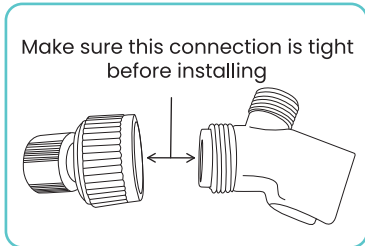


3. Install the mounting bracket to the existing water pipe.

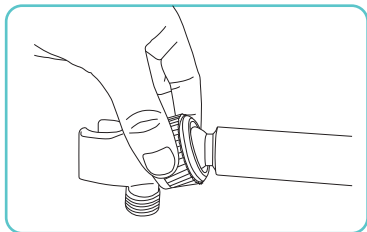

4. Connect the end of the shower hose to the holder bracket. For a tighter seal, it's recommended to add 2-3 layers of thread tape to the bottom end of the mounting bracket.

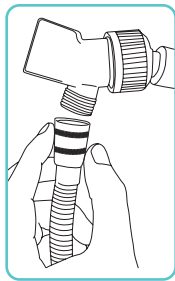

5. Attach shower head to the hose.

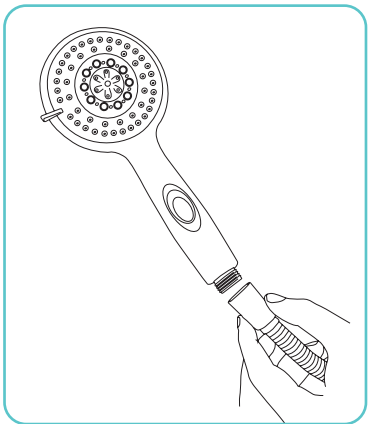

6. Make sure **all** connections are tightened down and test installation with **all** spray settings before use.

vive®

Distributed by

vive  
health