




## Fitting Protocol

Silhouette Sling  
Models 25015, 25020

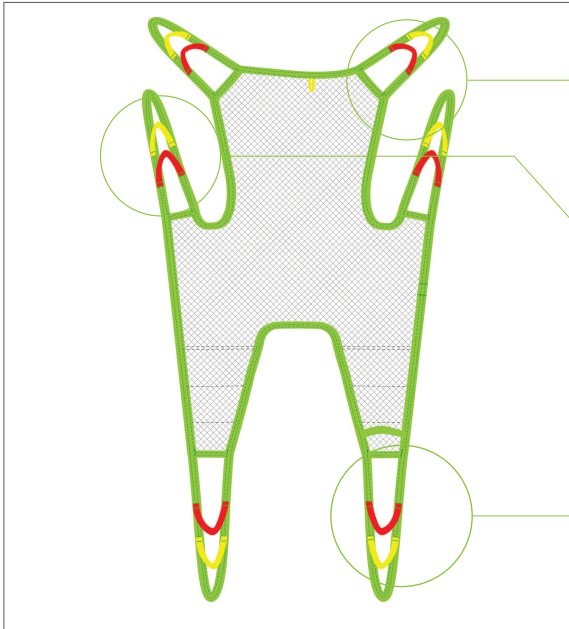

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops


- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

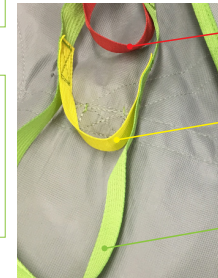

- Red Strap**  
Upright Position  
Least hip angle
- Yellow Strap**  
Semi-Recumbant  
Mid-hip angle
- Green Strap**  
Increased Recline  
Less hip flex and  
greatest angle

Match colours on both sides of the patient for a 90 degree angle in the hips.

**Maximum User Weight: 250kg/551lb**

- Model 25015 - Polyester Net**
- Model 25020 - Trevira**

Transfer Situation:

- To/From a Bed
- To/From a Chair
- Showering (25045 only)

Available sizes:

Indicate size required

- JS
- JM
- S
- M
- L
- XL

Leg Placement:

- Divided relaxed position
- Open position
- Divided position
- Closed position

Suspension:

- 2-Point
  - 4-Point
  - 3-Point
- Not available for this sling**

Accessories required:

- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large
- 25102 Reinforced Leg Support - One Size
- 50866 Upholstered Cover for Hanger Bar  
**Do not use in wet environments.**

Patient Information:

Name

Facility  Room

Lift Used  Hanger Bar Used

The Silhouette Sling is a comfortable sling without reinforcements, handles or padding. It adapts to the patient's body to offer good support. It can be used in most lifting situations and its narrow design makes it an optimal choice for patients who are smaller in stature. The Silhouette Sling is available in Polyester Net and Trevira fabric, both of which can be left behind the patient for a short period of time after the transfer is complete.

### Additional Instructions

To staff/caregiver:


To patient:


Fitting Protocol Completed by:

Name		
Date	Position/Title	



**Important!** This fitting protocol does not replace the user manual. Read all instructions before use!