

The Silhouette Sling is a comfortable sling without reinforcements, handles or padding. It adapts to the patient's body to offer good support for the entire body. The sling can be used in most lifting situations and is especially suitable for transferring to and from wheelchairs, molded seats or seating shells.

Two Fabric Choices

The Silhouette Sling is available in either Polyester Net or Trevira, both of which allow moisture and heat to transfer efficiently from the patient. These slings can be left behind the patient for a short period of time after the transfer is complete. The Polyester Net fabric is flexible, airy and dries quickly which makes it suitable for use in bathing environments.

Narrow Overall Width

The Silhouette Sling has a narrow overall width which makes it an ideal option for users who are smaller in stature. The Silhouette Sling can be used for most lifting situations with a 2-point, 3-point or 4-point suspension and is suitable for patients with

poor trunk stability. The Silhouette Sling is available in sizes Junior Small to Extra Large, accommodating a wide assortment of users.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180-*



Leg Padding and Supports

Ideal for users with sensitive skin or pain. Part no.: 21146-S, M, L Reinforced supports. One size. Part no. 25102



Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866



Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 [2- or 3-Point - Quick Release], 55619 [4-Point - Attach to leg hook]

Lifting Situations



Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



Use to/from a chair


Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



Use for showering


Only Polyester Net fabric can be used for this application.

Care and Features

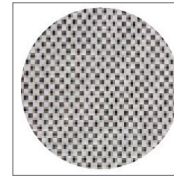


Polyest Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering.

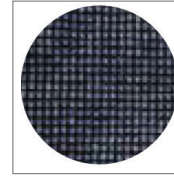


Trevira

Flame retardant, breathable material that does not promote skin breakdown.

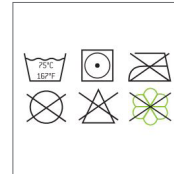

Care


Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

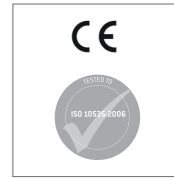

Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

Technical Specifications

| Part Number | Size | Colour | A | B | C | |
|----------------------|----------------|---------------|----|---------------|--------------|---------------|
| Polyester Net | Trevira | | | | | |
| 25015-JS | 25020-JS | Junior Small | JS | 18cm / 7.1" | 62cm / 24.4" | 64cm / 25.2" |
| 25015-JM | 25020-JM | Junior Medium | JM | 22cm / 8.7" | 67cm / 26.4" | 71cm / 28" |
| 25015-S | 25020-S | Small | S | 26cm / 10.2" | 77cm / 30.3" | 79cm / 31.1" |
| 25015-M | 25020-M | Medium | M | 29cm / 11.4" | 87cm / 34.2" | 90cm / 35.4" |
| 25015-L | 25020-L | Large | L | 35cm / 13.8" | 93cm / 36.6" | 96cm / 37.8" |
| 25015-XL | 25020-XL | Extra Large | XL | 40 cm / 15.8" | 97cm / 38.2" | 106cm / 41.8" |
| Maximum User Weight | 250kg/551lb | | | | | |

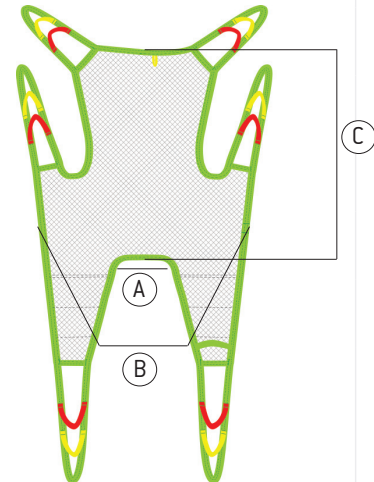

* Available in assorted lengths: 10cm/3.9" (21180-10), 15cm/5.9" (21180-15), 20cm/7.8" (21180-20), 25cm/9.8" (21180-25), 35cm/13.7" (21180-35)

This product is manufactured in accordance with all relevant ISO standards.

