




Fitting Protocol

Sit Sling
Model 25030

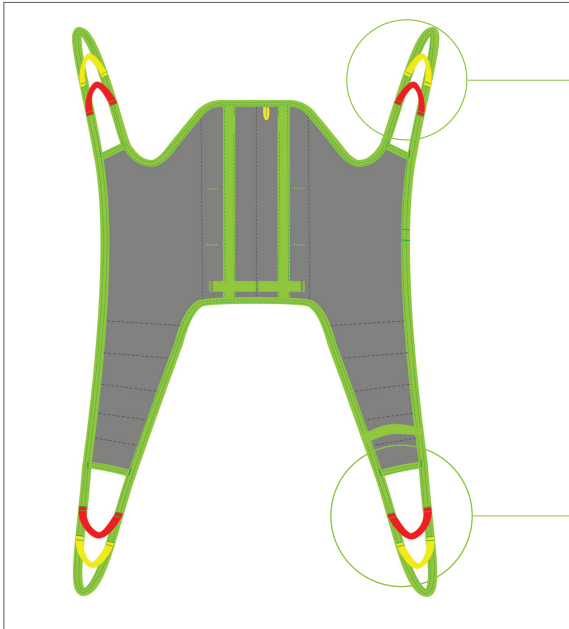


- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

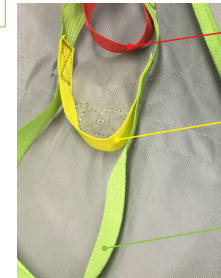

- Red Strap**
Upright Position
Least hip angle
- Yellow Strap**
Semi-Recumbant
Mid-hip angle
- Green Strap**
Increased Recline
Less hip flex and greatest angle

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 250kg/551lb

Model 25030 - Polyester

Transfer Situation:

- To/From a Bed
- To/From a Chair

Leg Placement:

- Divided relaxed position
- Open position
- Divided position
- Closed position

Suspension:

- 2-Point
 - 4-Point
 - 3-Point
- Not available for this sling**

Available sizes:

Indicate size required

- JS
- JM
- S
- M
- L
- XL

Accessories required:

- 21146 Leg Padding - Small
 - 21146 Leg Padding - Medium
 - 21146 Leg Padding - Large
 - 25102 Reinforced Leg Support - One Size
 - 50866 Upholstered Cover for Hanger Bar
 - 25145.* Waist Belt (Velcro)
 - 25146.* Waist Belt (Safety Buckle)
- | | |
|----------------------|---------------------|
| *Junior Sizes | *Adult Sizes |
| 40B (40cm/15.7") | 60 (60cm/23.6") |
| 50B (50cm/19.7") | 70 (70cm/27.6") |
| 50B (50cm/19.7") | 80 (80cm/31.5") |
| 60B (60cm/23.6") | 90 (90cm/34.4") |
| | 100* (100cm/39.4") |
| | 110 (110cm/43.3") |
| | 120 (120cm/47.3") |
| | 130 (130cm/51.2") |
- *only for Model #25146

Patient Information:

Name

Facility Room

Lift Used Hanger Bar Used

The Sit Sling is a shoulder high model with divided and reinforced leg parts as well as a reinforced back. It is designed to give the patient extra support and is a good solution for those users with poor trunk stability. Made of Polyester fabric, it is fast and simple to apply and can be used for most lifting situations. The Sit Sling creates an upright seating position which makes it ideal for proper positioning when transferring into a chair. The Sit Sling can be used with or without a back belt.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name		
Date	Position/Title	



Important! This fitting protocol does not replace the user manual. Read all instructions before use!