




The Sit Sling is a shoulder height model with divided and reinforced leg parts as well as a reinforced back. It is designed to give the patient extra support and is a good solution for those users with poor trunk stability.

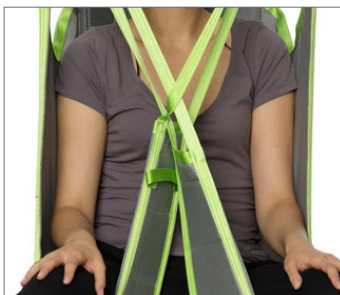

Versatile

The Sit Sling is made of a Polyester fabric which makes sling application fast and simple. Because of its easy application, this sling can be used for most lifting situations. The sling design creates an upright seating that ensures proper positioning when the patient is placed in a chair.

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.



Extra Support

The Sit Sling features a reinforced back and leg parts that offer additional support throughout the back area for those users with poor trunk stability. The Sit Sling can be used with or without an optional waist belt that is available in an assortment of sizes with a Velcro or buckle closure.

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no. 21180.*



Waist Belts

Velcro or buckle enclosure. Available in Junior and Adult sizing. Velcro Part no. 25145.** Buckle Part no. 25146-***



Upholstered Cover

Protects user from hanger bar impact. Part no. 50866



Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 [2- or 3-Point - Quick Release], 55619 [4-Point - Attach to leg hook]

Lifting Situations



Use to/from a bed


Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



Use to/from a chair


Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.

Care and Features

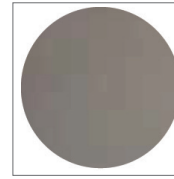


Polyester

Durable, low-friction, easy to clean, apply and remove.

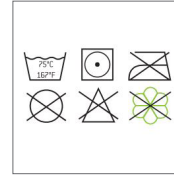

Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

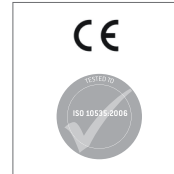


Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

This product is manufactured in accordance with all relevant ISO standards.



Technical Specifications

Part Number	Size	Colour	A	B	C
25030-JS	Junior Small	JS	22cm / 8.7"	56cm / 22"	53cm / 22.9"
25030-JM	Junior Medium	JM	28cm / 11"	63cm / 24.8"	58cm / 22.8"
25030-S	Small	S	35cm / 13.8"	76cm / 29.9"	62cm / 24.4"
25030-M	Medium	M	40cm / 15.7"	86cm / 33.9"	62cm / 24.4"
25030-L	Large	L	44cm / 17.3"	98cm / 38.6"	63cm / 24.8"
25030-XL	Extra Large	XL	48 cm / 18.9"	102cm / 40.2"	64cm / 25.2"
Maximum User Weight	250kg/551lb				

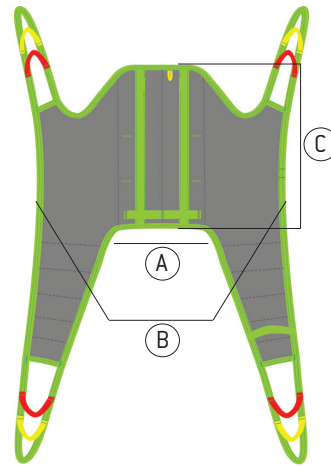

* Available in assorted lengths: 10cm/3.9" [21180-10], 15cm/5.9" [21180-15], 20cm/7.8" [21180-20], 25cm/9.8" [21180-25], 35cm/13.7" [21180-35]

** Junior Sizes: 40cm/15.7" [25145-40B], 50cm/19.7" [25145-50B], 60cm/23.6" [25145-60B]; Adult Sizes: 60cm/23.6" [25145-60], 70cm/27.6" [25145-70], 80cm/31.5" [25145-80], 90cm/35.4" [25145-90], 110cm/43.3" [25145-110], 120cm/47.3" [25145-120]

*** Junior Sizes: 40cm/15.7" [25146-40B], 50cm/19.7" [25146-50B], 60cm/23.6" [25146-60B]; Adult Sizes: 60cm/23.6" [25146-60], 70cm/27.6" [25146-70], 80cm/31.5" [25146-80], 90cm/35.4" [25146-90], 100cm/39.4" [25146-100], 110cm/43.3" [25146-110], 120cm/47.3" [25146-120]