


Foot-Drop Boot

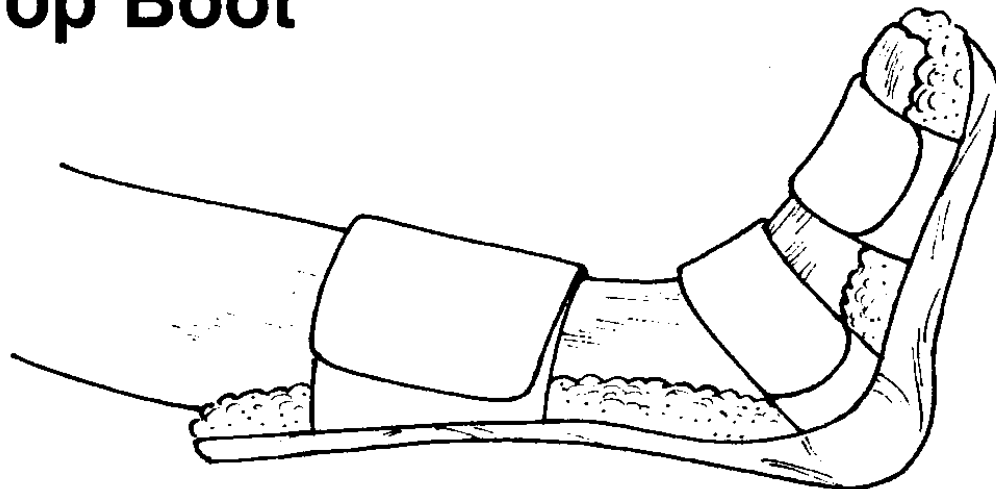

Description	Size	Shoe Size
Foot-Drop Boot	Small	Men's 5-7, Women's 3-6
Foot-Drop Boot	Medium	Men's 7-9, Women's 6-9
Foot-Drop Boot	Large	Men's 9 -12, Women's 9-11

Purpose

Designed with a supportive plastic inner splint that controls plantar flexion to reduce foot drop. Patient's heel is suspended above a molded cavity for maximum pressure reduction. Synthetic sheepskin liner adds comfort while providing protection against skin shear.

Application

1. Select proper size for patient. See size chart above.
2. Place patient's foot in boot so that heel is suspended over cavity.
3. Secure Velcro[®] straps.

Laundering

Remove cover. Do not launder inner splint. The Foot-Drop Boot cover may be laundered and dried using standard white goods procedures. Do not wash or dry at temperatures above 180°F (82° C). The use of bleach is not recommended.