

# ABDUCTOR/CONTRACTION CUSHION







Reorder# 555060

## PURPOSE

A new design for leg abduction and contraction control. Allows patient to sleep comfortably on his side or back. Easy to apply. Velcro® straps keep this cushion properly positioned for maximum effectiveness.

## Application

1. Place leg separator between patient's legs. Extend Velcro® straps to left and right sides. (See illustration A)
2. Lie patient on side and place strap through D-ring. (See illustration B) Secure Velcro to strap surface. (See illustration C) Repeat on opposite side. Note: Straps should be snug enough to keep cushion securely in place.
3. Check behind patient's knees to make certain that cushion is properly positioned. (See illustration D)

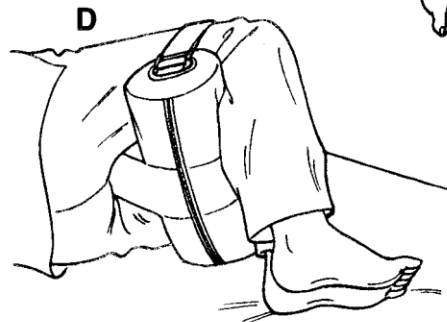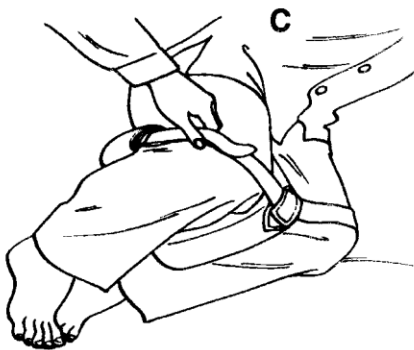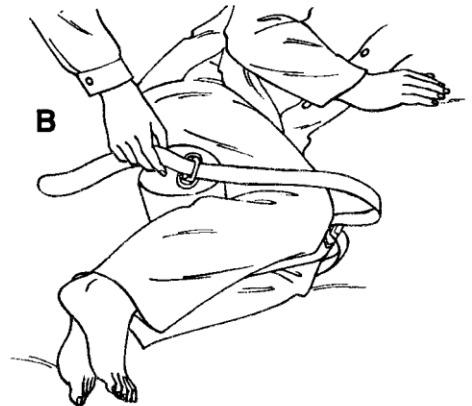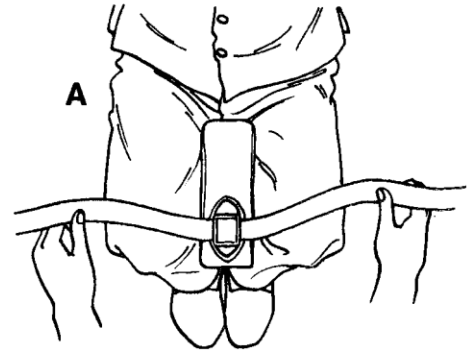

## Easy Care

The removable cloth cover may be machine laundered and dried. The vinyl cushion cover cleans with soap and water.