

HEAVY DUTY SKI GLIDES AND COVERS

QUICK START GUIDE


OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Heavy Duty Ski Glides and Covers as easy as possible. Check out the included link and QR code to help you through the process.

USER INSTRUCTIONS

Note: For walker without front wheels, install walker glides on front 2 legs. For walker with front wheels, install walker glides in the rear 2 legs.

1. Flip walker over and remove the front or rear two rubber feet.
2. Insert glides on each of the front or back legs. The arrows on the glide should be pointing towards the direction of forward movement.

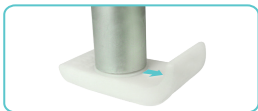

3. Use Philips screw driver to expand or retract diameter of insert. Tighten screw until glides are secured to the walker.



COVERS

Place the included gray covers over each glide if using on wooden, laminated, or tiled floors. Gray covers may be hand washed in cold water and air dried.

HOW TO REMOVE GLIDES

To remove glides simply loosen screw until glides are loose enough to come off. Remove and store away.

WARNING

These glides are designed to reduce friction, movement may feel slightly different for a first time user.

vive®

Distributed by

vive
health

280122