


LUMEX[®]

Sling Guide

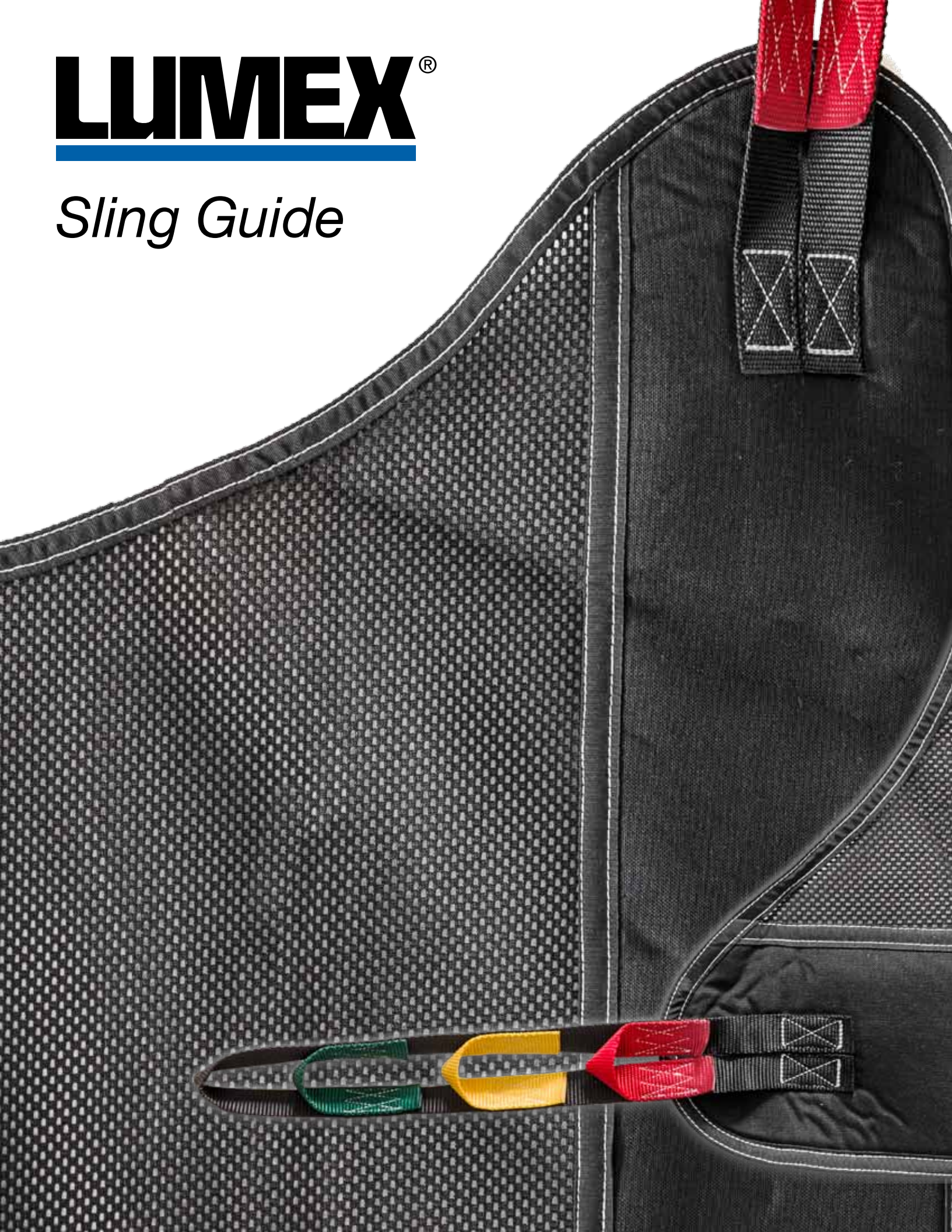

Full Body Slings

Full Body slings are the easiest sling to use while transferring a resident from a supine position (laying on ones' back). It can be used for transfers from a bed to a chair, wheelchair, or toilet, or from the floor to a bed. This sling style offers total head, neck, and thigh support. Full body slings should be used for residents who are totally or partially dependent, non-weight bearing, bariatric, or for those who have limited

head control. These are available in a solid fabric, mesh, or mesh with a commode opening. Mesh slings should be used for bathing. These slings are often left under the resident between lifts. Solid fabric slings should not be left under the resident for extended periods of time, as the fabric can retain body heat. LUMEX Full Body slings have four connection points and can be used with 2-, 4-, or 6-point spreader bars.



Style	Item	Size	Accent Color	Safe Working Load (lbs.)	Best Fits (lbs.)
Solid Fabric	F112	M	Red	450	100-195
	F113	L	Green	450	150-300
	F117	XL	Blue	450	200-400
Mesh	FM110	M	Red	450	100-195
	FM111	L	Green	450	150-300
	FM140	XXL	Yellow	600	290-600
Mesh with Commode Opening	FMC114	M	Red	450	100-195
	FMC115	L	Green	450	150-300
	FMC116	XL	Blue	450	200-400
	FMC141	XXL	Yellow	600	290-600