

Explanation of Symbols

Explicación de los símbolos

Explication des symboles

Erläuterung der Symbole

Spiegazione dei simboli

Explicação dos Símbolos

Verklaring van de symbolen

Symbolförläggningar

Условные обозначения

Επεξήγηση των συμβόλων

Rx Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).

Atención: En los EE.UU., las leyes federales solo autorizan la venta o uso de este dispositivo bajo receta médica (o de un profesional autorizado).

Attention : Selon la loi fédérale américaine, ce dispositif ne peut être vendu que sur prescription médicale.

Achtung: Laut Gesetz ist der Verkauf dieses Produkts in den USA nur auf ärztliche Anordnung gestattet.

Attenzione: La legge federale statunitense consente la vendita e l'uso di questo dispositivo esclusivamente da parte o per prescrizioni di un medico (o di personale debitamente autorizzato).

Atenção: A lei federal (dos Estados Unidos) só permite a venda ou utilização deste dispositivo a médicos ou sob receita destes (ou de outro profissional devidamente credenciado).

Opgelet: In de Verenigde Staten mag dit product uitsluitend door of op voorschrijf van een arts (of een daar toe bevoegd paramedisch beroepsbeoefenaar) worden verkocht.

Warning: Denna anordning får endast användas enligt läkare (behörig praktiker) föreskrift.

Предупреждение: Федеральный закон США разрешает продажу этого устройства только врачам (или другим медработникам с надлежащей лицензией) или по их предписанию.

Προσοχή: Η ομοιοποντική νομοθεσία των Η.Π.Α. περιορίζει την πώληση ή τη χρήση αυτής της συσκευής από ιατρό ή κατόπιν εντολής ιατρού.

Do not use if package is damaged
No utilizar si el envase está dañado
Ne pas utiliser si l'emballage est endommagé
Bei beschädigter Verpackung nicht verwenden
Non utilizzare se la confezione è danneggiata
Não utilizar se a embalagem estiver danificada
Niet gebruiken als de verpakking beschadigd is
Använd inte om förpackningen är skadad
Не использовать при нарушении целостности упаковки
Να μη χρησιμοποιείται αν η συσκευασία έχει υποστεί ζημιά

Contains or presence of natural rubber latex
Contiene goma de látex natural
Contient ou Présence de latex naturel
Gehalt oder Spuren von natürlichem Latexgummi
Contiene o presenta lattice di gomma naturale
Contém ou apresenta borracha de látex natural
Bevat of aanwezigheid van natuurlijke latex
Består eller innehåller naturligt gummilatex
Содержит натуральный латекс или состоит из него
Περιέχει φυσικό λάτεξ ή γινεται φυσικό λάτεξ

SOFT BODY BELT

RX ONLY

SINGLE PATIENT USE

NON STERILE

NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION
Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal Soft Body Belt is intended to be prescribed by a properly licensed practitioner to provide a gentle reminder for patients to stay in bed or wheelchair to help prevent falls or getting up unassisted.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Always secure straps to a portion of the bed or wheelchair that moves with the patient. Do not secure straps to bed or gurney side rails.
- Always keep straps out of patients reach.
- Belt should be worn over patient's clothing.
- Make sure bed or gurney side rails are up while product is in use.
- Check the patient regularly to ensure circulation is not compromised.

PRECAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

CEINTURE CORPORELLE DOUCE

RÉSERVÉ AU PERSONNEL MÉDICAL

REUTILISABLE

NON STERILE

NON FAIRE EN LATEX NATUREL

INFORMATION IMPORTANTE

Avant utilisation, lire l'ensemble des instructions, avertissements et précautions. Une bonne application garantit le bon fonctionnement du produit.

Utiliser uniquement sur la personne à laquelle le dispositif a été prescrit par un professionnel de santé et uniquement avec l'utilisation pour lequel il a été conçu.

USAGE PRÉVU

La ceinture corporelle douce DeRoyal, prescrite par un praticien agréé, est conçue pour rappeler gentiment au patient de rester au lit ou dans son fauteuil roulant, afin de lui éviter de tomber et de se redresser sans assistance.

CONTRE-INDICATIONS

- Ne pas utiliser sur un patient qui est ou qui pourrait devenir très agressif ou agité.
- Ne jamais utiliser sur un patient si le dispositif risque d'endommager un emplacement d'intraveineuse ou une blessure.

AVERTISSEMENTS

- Toute mauvaise application ou utilisation d'une entrave risque de blesser gravement ou de tuer le patient.
- Toujours attacher les sangles à la partie du lit ou du fauteuil roulant qui se déplace avec le patient. Ne pas attacher les sangles au lit ou aux barrières latérales du lit à roulettes.
- Toujours tenir les sangles hors de portée du patient.
- La ceinture doit être portée par-dessus les vêtements du patient.
- Veiller à ce que le lit ou les barrières latérales du lit à roulettes soient relevés lors de l'utilisation du produit.
- Vérifier périodiquement que la circulation du patient n'est pas entravée.

PRÉCAUTIONS

Ce dispositif doit être mis en place par un médecin (ou par un praticien agréé) qui connaît l'utilisation prévue du produit. Le médecin ou praticien doit fournir les instructions d'utilisation et les mesures de précaution aux autres praticiens ou prestataires de santé impliqués dans les soins du patient, ainsi qu'au patient. Consultez immédiatement votre médecin (ou

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Place belt on center of bed at patient's waistline.
NOTE: Patient should be sitting up while belt is being applied.
- Position patient on his or her back with belt against waist.
- Wrap the belt around the patient by feeding the narrow strap through slit in the belt in front of patient.
- Secure the straps to the bed with quick-release ties. Wrap the strap(s) around the anchor once. Take the loose end of the strap(s) and create a loop. Cross the loop over the other strap(s) that is wrapped around the anchor and feed it through opening as if tying a shoelace. Hold that loop and create another loop with the loose strap(s) and feed it through the first loop. Pull the final loop and the strap(s) attached to the cuff in opposite directions to form a quick release tie.
- To secure the straps to a wheelchair, take both straps around the patient and criss-cross them behind the wheelchair back. Attach the straps to the wheelchair tilt bars using quick release ties. Wrap the strap(s) around the anchor once. Take the loose end of the strap(s) and create a loop. Cross the loop over the other strap(s) that is wrapped around the anchor and feed it through opening as if tying a shoelace. Hold that loop and create another loop with the loose strap(s) and feed it through the first loop. Pull the final loop and the strap(s) attached to the cuff in opposite directions to form a quick release tie.
- Do not over-tighten. Two fingers should fit between the device and the patient to ensure that circulation is not compromised.

Application videos are available on www.deroyal.com.

SIZING

Wide

Size	Dimensions
Universal	56" L x 7" W
Small	42" L x 5 1/2" W
Medium	48" L x 7 1/2" W
Large	55" L x 7 1/2" W

Narrow

Size	Dimensions
Small	40" L x 4" W
Medium	47" L x 4" W
Large	55" L x 7 1/2" W
X-Large	66" L x 7 1/2" W

Dimensions

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

100 x 10 cm
120 x 10 cm
140 x 20 cm
170 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm

140 x 18 cm
106 x 14 cm
120 x 20 cm
140 x 20 cm