

Instruction Sheet: Steamroller Ramp 150045

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using this equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Rehabmart immediately at 1-800-827-8283.

MATERIALS LIST:

1 - Top Roller Section	1 - 7/32 Hex Wrench
1 - Bottom Roller Section	1 - Instruction Sheet
2 - Connector Boards	1 - Maintenance Checklist
16 - Bolts & Washers	

This item is made to use with the Steamroller Platform # 150045P

ASSEMBLY:

- First identify the top and bottom roller sections. They both have threaded inserts on one end where they connect. The bottom section has "feet" on one end as seen in figure 2. The top section has a "hook" on one end that connects to the rung of the platform ladder.
- Put a flat washer on each of the bolts.
- Position the top and bottom with the foot of the bottom and the open end of the hook on the top section turned the same direction.
- Position one of the connector boards on the bottom of the top section with the curved end next to the nut holding the roller as shown in figure 1. Start a bolt into each of the four threaded inserts. Do not tighten the bolts at this time.
- Position the top of the bottom section as shown in figure one and start four bolts into the threaded inserts. Using the hex wrench provided, tighten all bolts securely.
- Attach the connector board on the opposite side in the same manner and tighten all bolts securely.
- Attach the top end with the hooks to one of the rungs of the platform as shown in figure 2.



Figure 1

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the maintenance checklist enclosed.

- Check wood for splinters. If splinters are found, sand smooth and refinish with lacquer.
- Check for loose bolts. If loose bolts are found, tighten them securely.

STORAGE PROCEDURES:

- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Wipe with a damp cloth using soap and water or disinfectant. Dry before using.

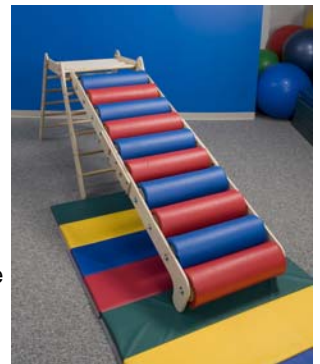


Figure 2

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. This equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all

If you have any questions or comments, please call
Rehabmart at 1-800-827-8283.