

# Instruction Sheet: Steamroller Platform 150045P

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. The Instruction Sheet must be read and familiar to anyone who is using this equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Rehabmart immediately at 1-800-827-8283.**

## MATERIALS LIST:

1 - Platform	1 - 7/32 Hex Wrench
2 - Ladders	1 - Instruction Sheet
8 - Bolts & Washers	1 - Maintenance Checklist

## ASSEMBLY:

- Put a flat washer on each of the eight bolts.
- Align the holes in the platform top with the threaded inserts in the ladder and start a bolt into the threaded inserts as shown in figure 1. Do not tighten the bolts until you have all of the bolts started.
- Using the 7/32 hex wrench provided, tighten all of the bolts securely.

## MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the maintenance checklist enclosed.

- Check wood for splinters. If splinters are found, sand smooth and refinish with lacquer.
- Check for loose bolts. If loose bolts are found, tighten them securely.

## STORAGE PROCEDURES:

- Store where not accessible by clients without proper supervision.

## CLEANING INSTRUCTIONS:

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Wipe with a damp cloth using soap and water or disinfectant. Dry before using.



Figure 1



Figure 2

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. This equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all

If you have any questions or comments, please call  
Rehabmart at 1-800-827-8283.