
# Quick Reference Guide

## Steady Aid Series™

tollos™

### ① Hand Controller / Control Box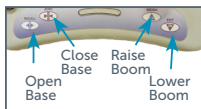

### ② iQ Technology

Digital readout scrolls through critical lift and service data every 7.5 seconds:

TTL CYCL / HRS Total Lift Cycles / Hrs  
 CHARGES Total battery charges  
 CHARGE PERC Current level of charge in battery  
 LOW CHRNG Aggregate number of very deep battery discharge  
 SERVC IN Number of lift cycles until service required  
 SERV CHK Indicates switch is stuck on hand control or touch pad

### ③ Emergency Stop ON/OFF

Push red emergency stop button in, located on control box, to turn OFF. To turn ON, press up/down or leg widening button (lift goes into sleep mode quickly). If red button is pressed in, it first needs to be popped out by gently pulling and turning clockwise. Do not use red button to turn off except in an emergency.

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### ④ Emergency Lowering

Pulling upward on red emergency lowering control initiates the lowering process.

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### ⑤ Locking Wheels

Lock wheels on lift when lifting patients, and unlock when lowering. For lifting or lowering always lock wheels on bed or wheelchair.

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### ⑥ Harness Loops

Ensure the harness loops are attached to the boom harness attach points.

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### ⑦ Shin Pad Positioning Seated Lifting

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**WARNING**  
 READ SLING MANUAL EXTREMELY IMPORTANT

### ⑧ Charging Batteries

- Charge the lift regularly after use
- The lift does not function during charging
- Lift will not overcharge
- DO NOT operate when charge light is ORANGE or RED as batteries permanently deplete

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## Simple Troubleshooting

- Check to ensure that:
- Emergency stop is not pressed in
- The battery is charged
- The hand control is connected properly
- Lift is not plugged in
- Turn on and depress any button on hand control
- Lift not at 6000 cycles

## Cleaning

When necessary, clean the lift with a moist cloth, using common hard surface cleaners or disinfectants.

## Inspection

- Check slings for good condition and size
- Check all nuts and bolts on lift for tightness
- Check actuator for free movement & no noise or vibration
- Check wheels lock, unlock & roll

## SAFETY WARNINGS

- READ Lift and Sling Manual before using
- DO NOT exceed lift or sling maximum weight
- DO NOT lift on angle. Keep patient under lift
- VISUALLY inspect sling and lift before each use for damage or rips to sling or loose bolts, etc on lift
- DO NOT tilt or lift on angle - lift may tip over
- ENSURE sling is correct size and type to hold patient; lift patient several inches over surface to test.
- SLING handles do not support weight
- LOCK all bed or chair wheels before lifting
- CHECK all bolts are tight, including actuator
- ATTACH harness fully and test belt

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