

HEAVY DUTY HINGED KNEE BRACE

SUP3032

1. Unclip buckles.



2. Grab brace from the side handles and slide your foot through.



3. Slide up to the knee and position the opening onto the patella.



4. Fasten the straps from bottom to top.



QUICK START GUIDE

vive

5. The straps may be cut to size if needed: remove the y-strap hook and loop, cut, then reattach.



CARE INSTRUCTIONS

We recommend removing the two hinged splints before washing. Hand wash with cold water and mild detergent. Hang dry.

WARNINGS

- Consult your doctor before use.
- Do not continue use if pain worsens.
- Do not overtighten.
- Do not machine wash or dry.