

AbleNet[®]

QUICK START GUIDE


SuperTalker™ FT

speech device

To get started:

1. Add four alkaline AA batteries to device (batteries not included).
2. Select and attach message grid for 1, 2, 4, or 8 message locations.
3. Turn device on.
4. Select a level and grid using the controls on the spine of the device.
5. Press and hold the record button until the record light activates, then release.
6. Press and hold the desired message location, begin speaking when you hear a beep. Release when finished.
7. Press and release the record button to exit record mode.
8. Your device is now ready to use.

3 Device Overview

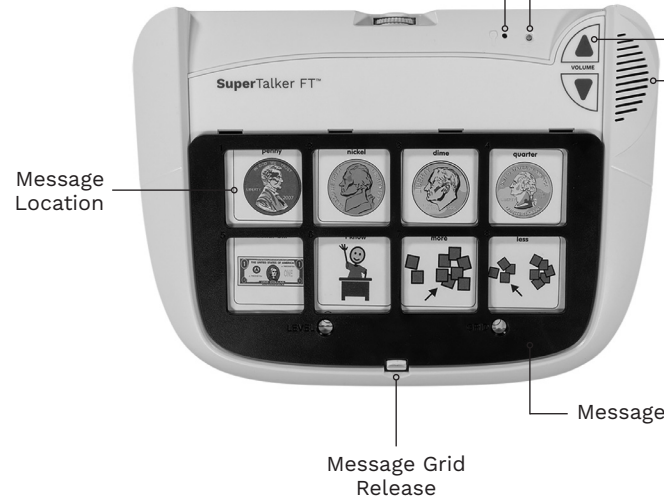

Microphone

Recording Indicator Light

Volume Control Buttons

Sound Output Speaker

Battery Compartment

