

vive®



SWIVEL SHOWER CHAIR

Owner's Manual
LVA2088


OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Swivel Shower Chair as easy as possible.

1

WHAT'S INCLUDED

- (A) 1x Seat Base
- (B) 4x Legs
- (C) 2x Armrests
- (D) 1x Backrest

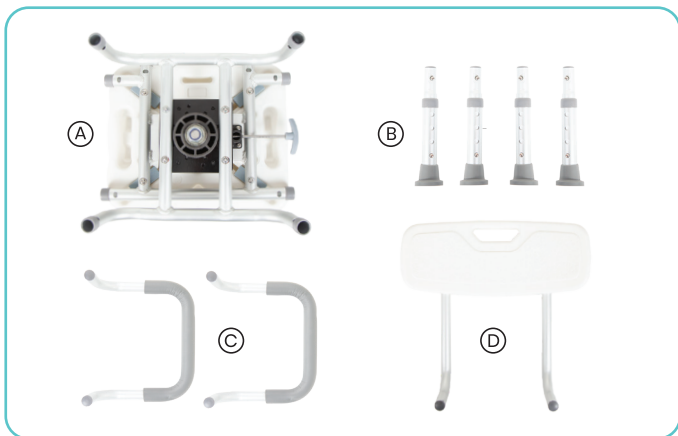

ASSEMBLY INSTRUCTIONS

Step 1

Take Seat Base (A) and flip it so it lays flat with the legs face up. Insert all four Adjustable Legs (B) onto the legs of the base so that the pins on each clip into place.



Step 2

Flip Seat Base (A) over so it is now sitting on its legs. Place Armrest (C) into the side slots on the chair so that the pins clip into place locking them onto the base. Repeat this step for the other side of the chair.



Step 3

Clip Backrest ④ into the back of the chair so it locks into place.



Step 4

Adjust the feet of the chair to fit your desired height. Make sure that the numbered position is the same on all four feet so the chair remains balanced.



CARE INSTRUCTIONS

Best to wipe dry after each use.

WARNINGS

- 300lbs weight capacity.
- Armrests are for resting your arms and providing guided support to raise yourself up, never place your full bodyweight on the armrests.