AbleNet®

TalkingBrix™ 2

www.ablenetinc.com
(800) 322-0956

Message Recording

1. On back of TalkingBrix, move the switch to REC
2. Press and hold colored switch top
3. Begin speaking for up to 10-seconds when light starts blinking
4. Release colored switch top when finished
5. On back of TalkingBrix, move switch from REC to ON to start using

A picture symbol up to 1.25-in / 32-mm can be used with TalkingBrix. Lift clear snap switch cap off of colored switch top to insert picture symbol. Re-attach clear snap cap to hold picture symbol on TalkingBrix.

Multi-Message Communication

You can connect multiple TalkingBrix together to create a multi-message communication device. Connect using the tabs and slots found on the side of each TalkingBrix.

Mounting

TalkingBrix include internal magnets for mounting. Or, use hook-and-loop fastener on the back of TalkingBrix.

Charging

When TalkingBrix has a low battery, the light will blink red three times after message playback. Attach TalkingBrix to the included wall charger. The light will be solid red when charging and solid green when fully charged. TalkingBrix cannot be used while charging.

Warranty and Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
(800) 322-0956
www.ablenetinc.com

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