




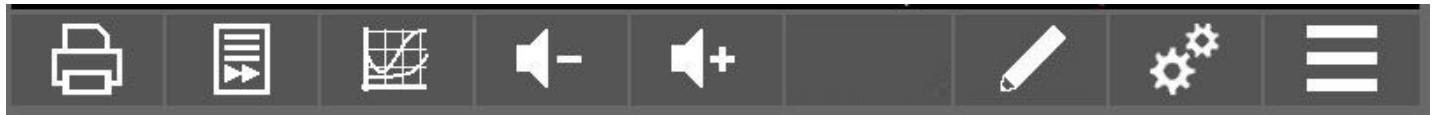
Team3 USA

Fetal Monitor

Switching the unit ON

- 1 Touch and hold the On/Off button for 2 seconds. (On Team 3 Front Panel). 
- 2 Check the printer: Is there sufficient paper for the monitoring session?
- 3 Connect transducers. Display shows transducer connection

Control Bar



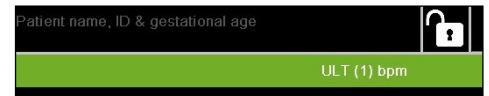
Printer On/Off Paper Feed Numbers or Graph Volume Down Volume Up Notes * Settings Menu *

(* Depending on model / operating mode)



- 1 Touch any of the keys above to operate that function.

Entering Patient Details

- 1 Touch and hold the box in the top left corner of the screen and a Patient Details dialogue appears.
- 2 Enter Patient Name using the popup keyboard.
- 3 Touch the Patient ID field and type data as required.
- 4 Touch the Gestational Age field to bring up another dialogue. Use the up and down arrows to set the correct gestational age.
- 5 Touch to set Gestational Age.
- 6 Touch to set Patient Data.
- 7 Ensure patient data is entered before you print. It cannot be changed afterwards.



Monitoring FHR

- 1 Determine fetal lie and position.
- 2 Apply Gel to the green transducer face.
Position on abdomen so as to get optimal heart sound. Secure with belt.
- 3 Make sure the FHR is clear, and different from the maternal pulse taken at the mother's wrist.
FHR signal quality is shown by the flashing heart symbol.
- 4 Adjust the volume for the audio channel.  

Monitoring Uterine Activity

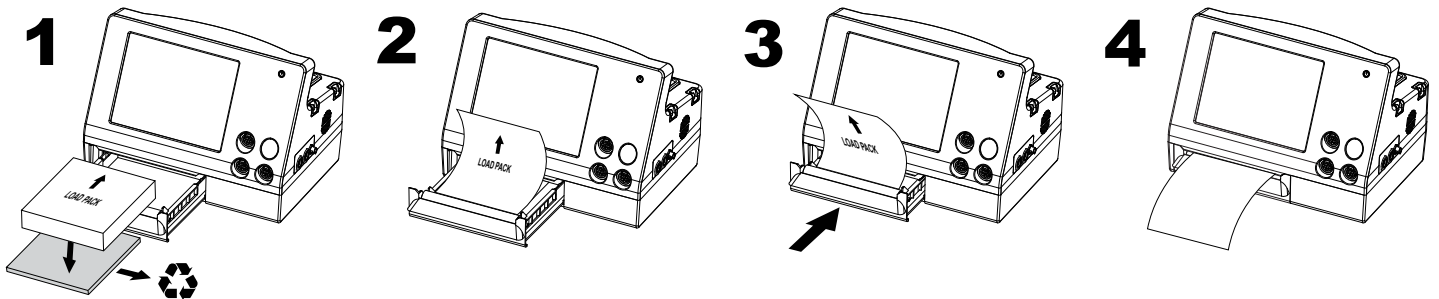
- 1 Position the pink transducer on the midline over the fundus of the uterus. Secure with belt.
- 2 Zero the Toco. Make sure the uterus is relaxed, and then touch and hold the screen in the pink TOCO Clinical Region.





Monitoring FHR for twins

- 1 Connect both green transducers to Team 3. Display shows transducer connection.
- 2 Apply Gel to the transducer faces, and position them so as to get optimal heart sounds.
- 3 Ensure you are not getting two signals from the same twin. If you are, Team 3 will warn that similar FHRs have been detected.
- 4 Secure the transducers with belts.
- 5 To change the Audio channel, touch and hold the screen in the FHR Display Region of relevant trace.

Loading paper







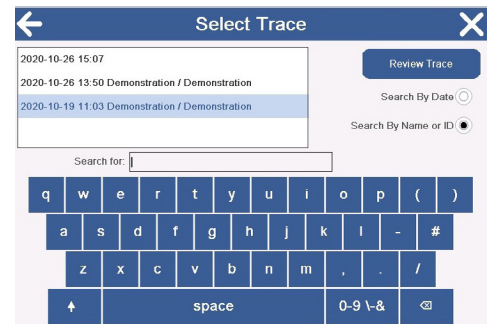
Printing the trace

- 1 To start or stop printing *, touch Printer On/Off key  on the control bar.
- 2 To advance the paper, touch Paper Feed .

* Refer to the Settings Menu in the instructions for Use (Settings > System > Recording Limits) for details on setting a automatic printing stop time.


Store Trace (PDF) to USB Drive (8GB max) *

- 1 Select View Menu .
- 2 Review Trace .
- 3 Enter PIN code to enable access to stored traces (PIN code : 98765).
- 4 Select trace from list by using the arrow to scroll through the traces .
- 5 Insert USB into USB drive in the rear panel of the fetal monitor.
- 6 Select View Menu and then select Discharge Patient .
- 7 Remove USB from the fetal monitor to enable trace to be shared on a PC.



* Refer to Sonicaid Reporter Quick Reference Guide for details on uploading stored traces to the Sonicaid Reporter software package.

Switching off

- 1 Touch and hold the On/Off button, (On Team 3 Front Panel) , until 'Power' screen is displayed. Touch 'Power Off'.