



Core Products
INTERNATIONAL, INC.

800-365-3047 | www.coreproducts.com



Therapeutica Sleeping Pillow

Print page at 100% for proper sizing, do not scale to fit page. To verify this guide has been printed true to size, the measurement of the Measurement Legend when printed should be 1". If you are unable to print measuring guide, please use a ruler and follow the instructions below to find your correct pillow size.

1. Have someone assist in measuring you
2. Move your shirt collar, if it interferes
3. Stand with arms at your side and looking straight ahead
4. Hold the measuring guide or a ruler to the base of the neck and measure to the outside tip of the bony prominence at the shoulder. NOTE: Ignore any muscle mass beyond the bony prominence. Choose one size smaller if you measure between two sizes or sleep on a soft surface such as memory foam or pillow-top mattress.

Measure from the base of the neck to the outside tip of the bony prominence at the shoulder.

NOTE: ignore any muscle mass beyond the bony prominence. Choose one size smaller if you measure between two sizes or sleep on a soft surface such as memory foam or pillow-top mattress.



**SELECT
SIZE BELOW:**

PETITE

AVERAGE

LARGE

X-LARGE

PILLOW SIZE

Petite
Average
Large
Extra Large

SHOULDER LENGTH

4¼" - 5¼"
5¼" - 6¼"
6¼" - 7¼"
7¼" - Plus

Measuring with
measuring guide



Measuring with
ruler




Trying is believing! Once you have tried a **Therapeutica® Sleeping Pillow**, you will understand why we believe it is the most comfortable foam pillow for back and side sleeping! A restful night of sleep is waiting for you.

If you have any questions, please feel free to call
Customer Service Department at 1-800-365-3047

*Not recommended for children
under 6 years old unless directed by a
Health Care Professional.

Cut along the dotted line

1"

Measurement Legend