



# Wunzi™ Infant System

Wunzi – a customized body hug that improves trunk alignment and stabilizes the core for those critical infant growth months.

## Ease of Use

The elegant over-the-diaper design goes on in seconds. Closure tabs are embedded right onto the garments – no loose tabs to keep track of!

## Familiar Design = Improved Compliance

Inspired by the standard infant “onesie” clothing, the Wunzi offers caregivers and clinicians a simpler alternative to the two garment TheraTogs system.

## Three Systems in One

The Wunzi™ Infant System consists of the core-strengthening Wunzi garment set, plus size-appropriate limb cuffs and straps. It's a complete infant-sized alternative that combines the benefits of three systems - the Posture & Torso Alignment system, Full Body system, and Lower Extremity system – into one easy-to-use package.



### Indications for Use Include:

- Hypotonia • Diplegic or Hemiplegic • Cerebral Palsy • Torticollis • Athetosis
- Cerebellar Ataxia • BPI • Developmental Delay



**Babies build trunk and neck control and strength before they gain functional skills.**

*Trunk control problems that occur in infancy typically lead to long-term developmental deficits in limb use and movement.*

*Postural control acquisition is a necessary component of early intervention.*

*The Wunzi system is designed to help very young clients by increasing sensory input while improving core stability.*

*Like all TheraTogs systems, the Wunzi can support therapy goals both during and between therapy sessions, providing essential carry-over. In short: **Wear it, live it, learn it!***

*– Beverly Cusick, PT, MS, COF | TheraTogs Inventor*



# Wunzi™ Infant System

## Use the Wunzi to:

- ✓ Increase sensory awareness and information
- ✓ Train functioning trunk muscles in shorter state to improve postural alignment and control
- ✓ Limit excessive trunk motions
- ✓ Improve functioning alignment of hips and/or shoulders
- ✓ Gently stabilize one or both scapulae
- ✓ Improve respiration



(Fig. 1) Torso Length

## Measurements for Selecting Wunzi Size

**Torso Length** (in CENTIMETERS) (see Fig. 1) - Measure down from C-7 over the diaper and up to the child's navel. Be careful to wrap the tape gently under the diaper.

**Belly Girth** - Measure the infant's belly circumference at the navel (in cm).

Model No.	Model Name	Age	Weight	Torso Length	Belly Girth
GWZ25	Wunzi, XXSmall	0 - 3 mo.	3.2 - 5.4 kg	40 cm	40 cm
GWZ50	Wunzi, XSmall	3 - 6 mo.	5.4 - 7.7 kg	50 cm	44 cm
GWZ75	Wunzi, Small	6 - 12 mo.	7.7 - 8.2 kg	55 cm	46 cm
GWZLKS	Limb Kit, Small	0 - 12 mo.	Up to 8.2 kg		
GWZ100	Wunzi, Medium	12 - 18 mo.	8.2 - 9.5 kg	60 cm	50 cm
GWZ125	Wunzi, Large	18 - 24 mo.	9.5 - 11.8 kg	68 cm	54 cm
GWZLKL	Limb Kit, Large	12 - 24 mo.	Up to 11.8 kg		

Like all TheraTogs systems, the Wunzi is a comfortable, foam-lined, Velcro®-sensitive garment and strapping system that grips the skin.

The patented composite fabric is elasticized with a stretch bias that favors good postural alignment – just the thing for a growing baby. The Wunzi includes grip-lined elastic straps that operate like external muscles and ligaments to improve functioning body and proximal joint alignment in order to optimize motor development.

## How to Order

For ordering guidance visit [www.TheraTogs.com/Ordering](http://www.TheraTogs.com/Ordering)

## About TheraTogs

TheraTogs™ are FDA Class I medical devices intended to be issued by, and applied under the supervision of, a licensed healthcare practitioner engaged in neuromotor or sensorimotor retraining. TheraTogs systems are made of GoldTone™ - a proprietary and patented composite fabric with foam backing made of an aqueous-based elastomeric urethane. TogRite™ strapping consists of elastomeric strapping with an inert, silicone-based grip surface. TheraTogs components are not made with natural rubber latex.

