




Fitting Protocol

Toileting Sling
Model 25080, 25085, 25090

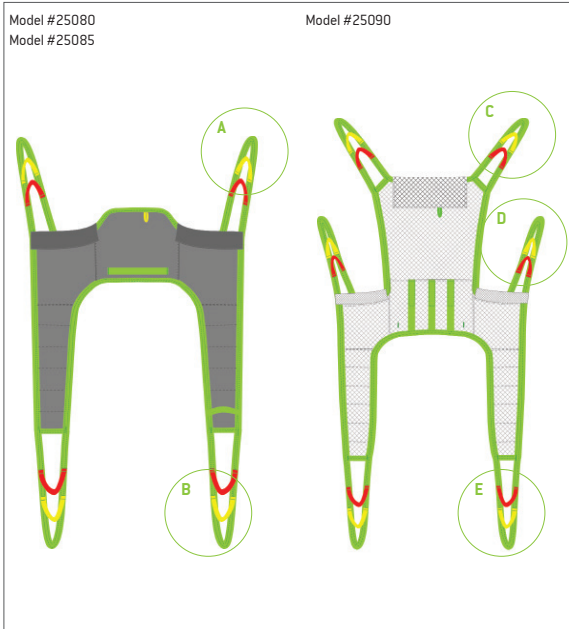

Sling Loop Letter: A B C D E

Green Strap
 Yellow Strap
 Red Strap

Extension Loops
 21180-10 (10cm/3.9")
 21180-15 (15cm/5.9")
 21180-20 (20cm/7.8")
 21180-25 (25cm/9.8")
 21180-35 (35cm/13.7")

Sling Loop Letter: A B C D E

Green Strap
 Yellow Strap
 Red Strap


Extension Loops
 21180-10 (10cm/3.9")
 21180-15 (15cm/5.9")
 21180-20 (20cm/7.8")
 21180-25 (25cm/9.8")
 21180-35 (35cm/13.7")

Sling Loop Letter: A B C D E

Green Strap
 Yellow Strap
 Red Strap

Extension Loops
 21180-10 (10cm/3.9")
 21180-15 (15cm/5.9")
 21180-20 (20cm/7.8")
 21180-25 (25cm/9.8")
 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

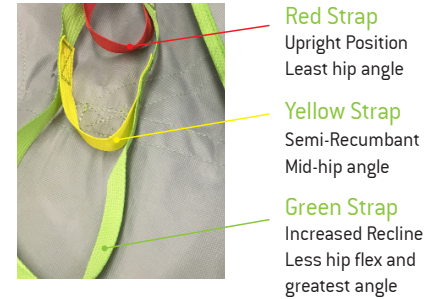

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 250kg/551lb

Model 25080 - Polyester

Model 25090 - Polyester Net

Model 25085 - Polyester Net

Transfer Situation:

To/From a Bed To/From a Chair Showering (25085/25090 only) Toileting (25085/25090 only)

Leg Placement:

Divided relaxed position Open position

Divided position Closed position

Suspension:

2-Point 4-Point

3-Point

Not available for this sling

Available sizes: Indicate size required

JM S M L XL

Accessories required:

21146 Leg Padding - Small
 21146 Leg Padding - Medium
 21146 Leg Padding - Large

25102 Reinforced Leg Support - One Size

50866 Upholstered Cover for Hanger Bar. **Do not use in wet environments.**

25145.*
Waist Belt (Velcro)

25146.*
Waist Belt (Safety Buckle)

*Junior Sizes	*Adult Sizes
-40B (40cm/15.7")	-60 (60cm/23.6")
-50B (50cm/19.7")	-70 (70cm/27.6")
-60B (60cm/23.6")	-80 (80cm/31.5")
	-90 (90cm/34.4")
	-100 [†] (100cm/39.4")
	-110 (110cm/43.3")
	-120 (120cm/47.3")
	-130 (130cm/51.2")

Size [eg. -60B]

[†]only for Model #25146

Patient Information:

Name

Facility Room

Lift Used Hanger Bar Used

Our Toileting Slings feature a wide, divided leg partition that enables fast and simple application for dressing and undressing during toileting or bathing. These slings are suitable for patients with poor trunk stability and feature padding under the arms for extra comfort and support. They are available in a low back model or a high back model with an adjustable neck support for increased head stability and are available in Polyester or Polyester Net for the low back model and Polyester Net for the high back model. All models can be used with an optional belt for added support.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name		
Date	Position/Title	



Important! This fitting protocol does not replace the user manual. Read all instructions before use!