


封面




TRIHEAR®
hear the connection

Clip Pro Pocket Hearing Amplifier

User Manual
Model: TH-L10M


Thank you for purchasing your new TH-L10M Clip Pro Pocket Hearing Amplifier. This user manual will provide you with all of the information you need to get started with your Clip Pro Pocket Hearing Amplifier. If you have any problems setting up or using your Clip Pro Pocket Hearing Amplifier, please contact us by email at: trihearing@outlook.com or scan the QR code to log in to our website: <https://www.trihear.com>

Package Contents

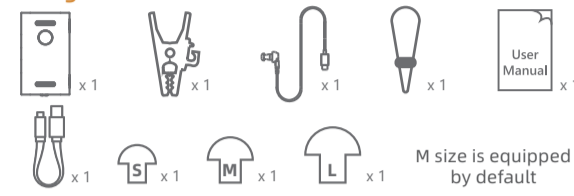

 x1 x1 x1 x1 x1 x1 x1
 M size is equipped by default

Product Overview

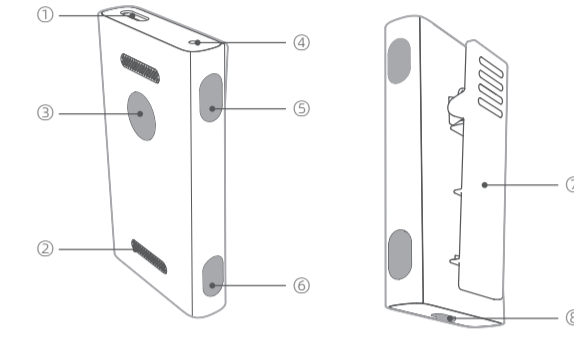

① USB-C Port ⑤ Volume +
 ② Mic ⑥ Volume -
 ③ Mode Switch ⑦ Back Clip
 ④ LED ⑧ Power On/Off

EN-01


Product Specifications

- Power input: 5 V $\overline{\text{---}}$ 1 A
- Frequency response range (Hz) : 200-6500
- Equivalent input noise (dB) : $\leq 18+3$
- Total harmonic distortion: $\leq 3\%+1\%$
- Maximum saturated sound pressure level OSPL90 (dB) : 130 ± 3
- High frequency average OSPL90 (dB) : 124 ± 4
- HFA Full of Gain(dB): 35 ± 5

How to use


1. Charging

Please charge the device with included USB-C Port Cable, the estimated charging time is 45 mins.
(*Power adapter not included, please use regular 5V DC adapter)
(We recommend you full charge the device before first use)

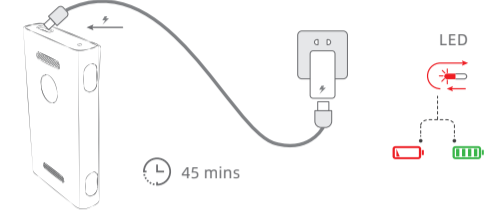

2. Connect Receiver


Please connect the pocket amplifier and receiver with included USB-C-2pin cable.

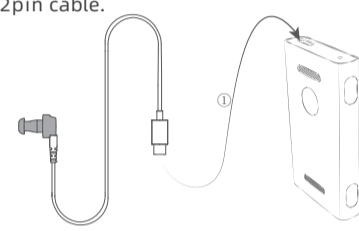

EN-02


3. Wearing

- Please choose suitable domes from S/M/L size (M size is equipped by default)
- Attach the domes to the receiver (no need if you choose M size)
- Put the receiver in the ear

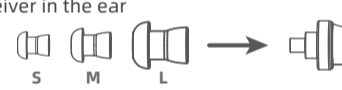


4. You could use either clip or lanyard when wearing

Use Clip

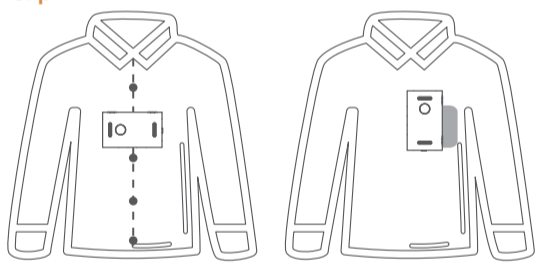


Use A Lanyard

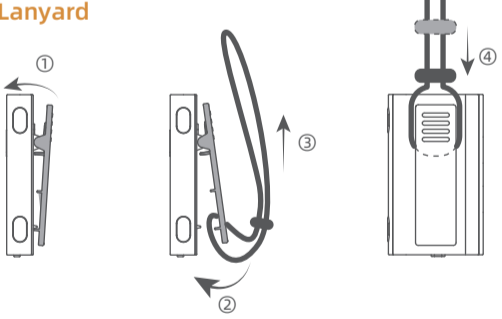

EN-03

5. Power On

Switch to 'on' to power on the device, LED will blink to indicate battery status




* x4 Blink 4 times Full Battery
 * x3 Blink 3 times Battery High
 * x2 Blink 2 times Battery Medium
 * x1 Blink 1 time Battery Low
 * x2 Blink 2 times Please Charge

Note:
Please wear receiver before turning on the device.

6. Volume Adjustment


Single click V+/V- button to adjust volume until you feel comfortable. Each time you click, there will be 'beep' tone, and 'double beep' tone when reach maximum/minimum volume.

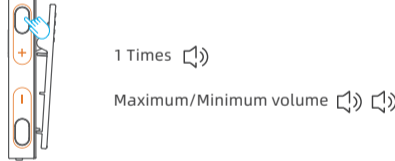

1 Times
 Maximum/Minimum volume

7. Mode Switch

Single click 'O' button to switch hearing mode.
* Please change to suitable mode according to your preference.

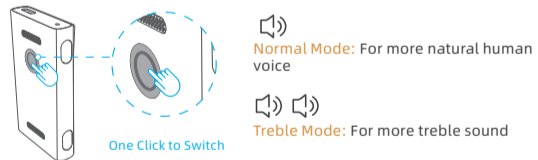

Normal Mode: For more natural human voice
 Treble Mode: For more treble sound
 One Click to Switch

EN-04

Note:
Kindly reminder that the hearing amplification will turn on automatically after powering on, and it will remember the volume and mode you use last time.

FAQ

Q: How long does it take to full charge the hearing aids?
A: It will full charge in 45mins.

Q: Could it be used with other USB-C earbuds or receiver?
A: No, you need to use included receiver and USB-C cable (the receiver is customized for the hearing impaired), in case of no sound or bad performance problems.

Q: Hearing amplification is not good.
A: Please adjust volume by clicking V+/V- button to your suitable volume.

Q: I could hear my own voice loudly.
A: You could adjust hearing amplifier position to make the Mic far from your mouth.

Q: I couldn't hear other people talking in noisy environment.
A: Please make sure Mic faces towards to the people you want to hear.

Q: No sound from the receiver.
A: Please check if the Mic is blocked, or the USB-C port is blocked, make sure the switch is slid to 'on'.

FCC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and
(2) This device must accept any interference received, including interference that may cause undesired operation.

EN-05

Safety Instructions

- Avoid dropping.
- Do not disassemble.
- Do not submerge in water.
- Avoid extreme temperature.
- Do not use the device outdoors during thunder storm.
- Use original or certified USB cables.
- Do not use any corrosive cleaner/oil to clean.
- Avoid using excessive volume for long time.
- Avoid thermal radiation, take off when near NMR, MRI, CT scanner, or other medical machines.
- Dispose old appliances according to local regulations.

Correct Disposal of This Product
(Waste Electrical & Electronic Equipment)

This Marking shown on the product or its literature, indicate that it should not be disposed with other household wastes at the end of its working life.

To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. Household user should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

Legal Disclaimer

Statements regarding dietary supplements have not been evaluated by the FDA and are not intended to diagnose, treat, cure, or prevent any disease or health condition.

EN-06

尺寸: 95x135mm

材质: 157g双铜 四色印刷 风琴折

注意: 黑色边框为裁切框 不印刷