

## Assembly instructions:

Remove the UPWalker Premium Lite v2 from the box and check the contents and the packaging for any damage or missing parts. This rollator is supplied in disassembled form.

Included in packaging:

• Main frame x1	• User manual x1
• Cup holder x1	• Personal Shopping bag x1
	• Deluxe bag x1
• Backrest x1	• Lights x2



2. Grab the handles and lift the frame upward until the upper frame is vertical to the base (refer to images).



3. Turn the orange lever so that the slot fits with the small locker, then turn the locker clockwise to lock the mechanism. Repeat this on the other side.



4. Press down from both sides of the seat to open the frame. A click will be heard once it is locked. ALWAYS ensure the seat is locked before sitting on it.



5. Turn the upper handle upward and lock the silver lever by turning it inward. Repeat this on the other side (refer to images).



6. At this stage, the UPWalker Premium Lite v2 should be unfolded as shown in the picture.



7. Locate the U clip – Adjust the soft plastic fitted piece, then clamp it onto the upper handle, insert the bolt of the cup holder through the 2 holes, lock it in twisting the knob on the other side; (See image below). The cup holder is attached to the product. Please note that the cupholder can be attached on either left or right side.



8. Open the two white small boxes and take out the LED lights. Firmly press each light into the metal holder, ensuring it is tightly attached and won't fall off easily (refer to images). Repeat this for the other side.



9. To add the personal shopping bag and attach one side of the hook onto the front seat tubing, then repeat on the other side (refer to images).



10. To add the deluxe bag, open the Velcro on one side, attach it to the upper handle, then repeat on the other side (refer to images).



11. Take the backrest with the logo facing the front of the UPWalker Premium Lite v2. Fit the slot on one side with the ridge on the frame and slide it down to the end. Repeat on the other side.



Your UPWalker Premium Lite v2 assembly is now complete.

