



# How to Use Guide

## Wheelchair Customized

### 1 Placing the SitnStand

**PRIOR TO PLACING THE SITN STAND, MAKE SURE THAT THE WHEELCHAIR BRAKES ARE LOCKED**

A. Place the SitnStand folded on the wheelchair - **Illustration A.**

B. Insert the back part / Air Supply area between the back and the seat of the wheelchair - **Illustration B.**

C. Place the air supply unit's slings on the wheelchair's handles and tighten them up - **Illustration C.**

D. When not in use we recommend you attach the remote control to a velcro strip that will be attached to the wheelchair push handles - **Illustrations D1 + D2.**

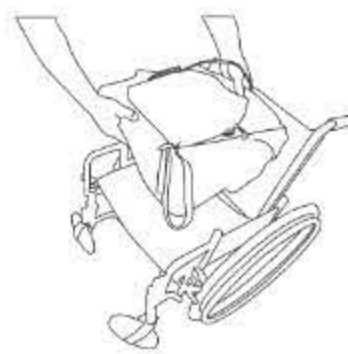


Illustration A



Illustration B

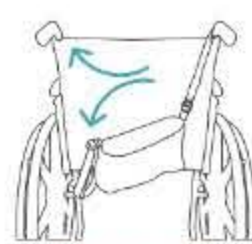


Illustration C

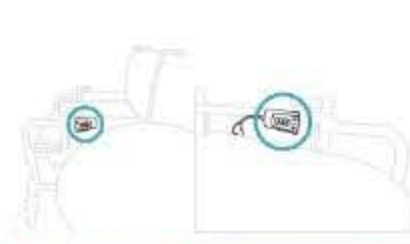


Illustration D1

Illustration D2

### 2 Sitting Down

**LOCK THE WHEELCHAIR BRAKES BEFORE SITTING DOWN!**


A. Sit Comfortably in the center.

B. Start deflating by pressing the "Down" arrow button.

C. You Can Now Sit Down Comfortably.

**\*\*NOTE:**

When using the battery for the first time it should be charged for at least 12 hours.

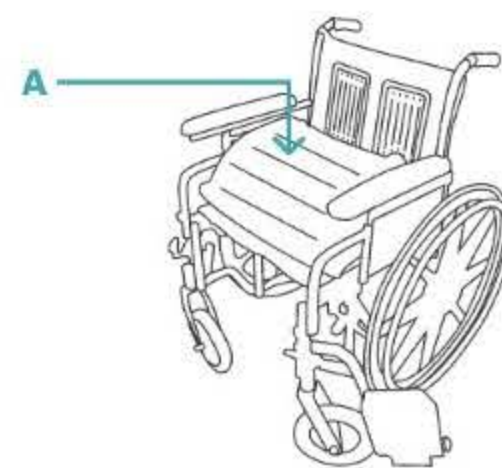

### 3 Standing Up

**LOCK THE WHEELCHAIR BRAKES BEFORE SITTING DOWN!**

A. Take your feet off the wheelchair paddles and place them on the floor, close to the chair.

B. Sit upright.

C. Spread your feet.

D. Inflate the cushion by pressing the "Up" button.

**E. Now you can easily Stand Up.**

**\*\*NOTE:**


In case of cognitive impairment the SitnStand should only be operated by a caregiver.



### 4 Get to Know the Remote Control

**\*\*NOTE:**

Standby mode- flashes slowly (one flash/sec) | Working mode - flashes quickly (5 flashes/sec) | Malfunction reset - press the two control buttons simultaneously for 10 secs. If that doesn't help, please contact us at - [info@sitnstand.com](mailto:info@sitnstand.com)

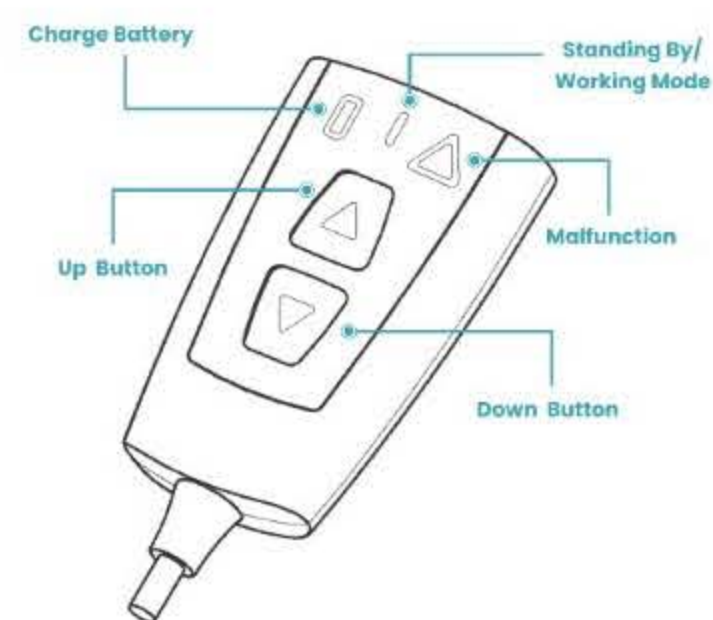

### 5 Standing Up

A. Remove battery by sliding it towards you.

B. Connect battery to the charger as illustrated.

C. When the "Red" light on the charger changes to "Green", the battery is fully charged.

D. Slide the charged battery into its place.

**\*\*NOTE:**

When using the battery for the first time, it should be charged for at least 12 hours.

