



Stackable Bath Step

Weight Capacity: 300lbs



INSTRUCTIONS INSIDE

The MOBB Stackable Bath Step is designed to make getting in and out of the bath or tub easier. The MOBB Stackable Bath Step features a large flat slip proof stepping surface with built-in non-slip pads. Two or more steps can be combined to raise the height of the steps by following the instructions below.


instructions

To combine both steps , follow the instructions below.

1. Place bath step on flat surface.
2. Using a coin, remove each of the 4 white circles located on the top of the bath step. Store these for future use.



3. Place second bath step on top of first, lining up prongs on underside of step with receptacle holes. Press down firmly to secure in place.

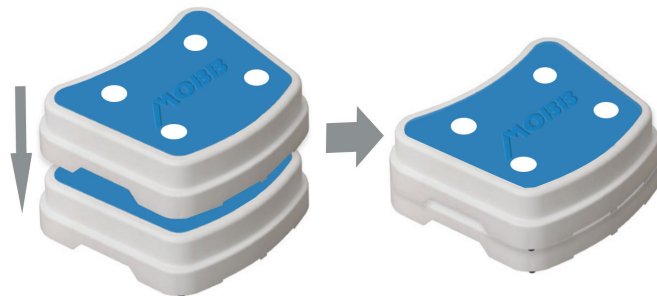

LIFETIME LIMITED WARRANTY

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer. MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.