





The Ultimate Toilet Safety Frame: MHUTSF Installation Guide

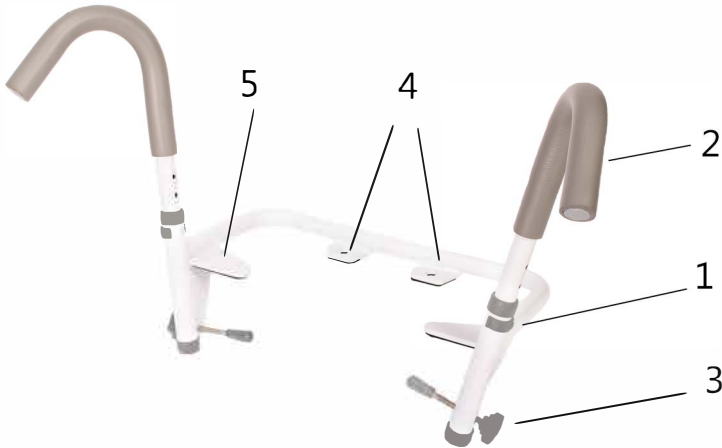

Installation

- 1) Remove existing toilet seat.
- 2) Make sure tension knobs (3) are fully unscrewed.
- 3) Place Ultimate Toilet Safety attachment plate (4) over the existing holes on back of toilet.
- 4) If not using toilet seat riser, place toilet seat cover over attachment plate (4) and secure with new hardware.
- 5) Tighten tension knobs (3) *equally* on both sides of the toilet bowl.
- 6) Remove C-clips (1) and place hand supports (2) (making sure they are installed on correct sides {left and right}), to desired height and replace C-clips (1) back.

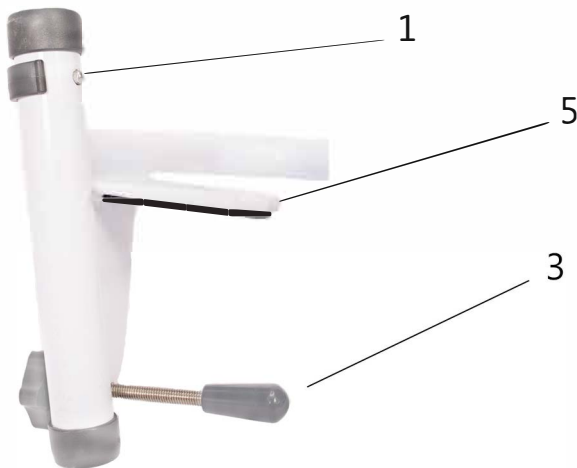

Note

Inspect the C-clips (1) and tension knobs (3) periodically to ensure they have not loosened.

1. C-clip
2. Hand support
3. Tension nob
4. Ultimate Frame attachment plates
5. Bowl support