



35207

**SELECTION
OPEN WRIST**

Intended use

Selection Open Wrist is intended to be used when the wrist needs off-loading in conditions such as rheumatic disorders or after healed fractures.

Indications:

Inflammations, work related strain injuries or when rheumatic and/or degenerative changes are present.

Contraindications

No known.

Adjustments

This product should only be altered by medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from polyester, nylon and elastan. Spoons are made from plastic. This product is not flame retardant.

Washing Instruction


Close all Velcro prior to laundering. Remove spoons before laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.



Instructions for use

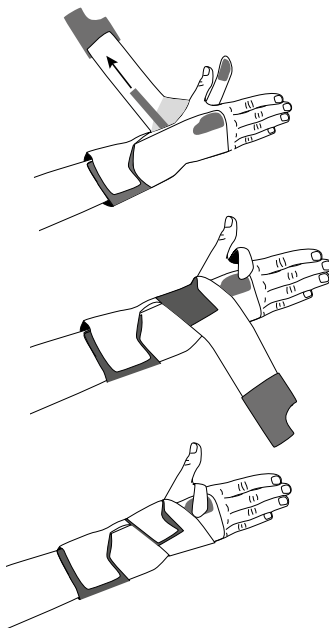

1. Check that the support is complete and without damage.
2. Open all Velcro straps and position the support with the splint inside the hand.
3. Secure the short upper Velcro strap and the thumb strap.
4. Then pull the long strap and wrap it around the wrist.
5. Adjust the short upper strap.
6. Adjust for comfort.



Item No.	Side	Size	Wrist circumference
35207 1010	Left	X-Small	< 11 cm
35207 1011	Left	Small	11 - 14 cm
35207 1012	Left	Medium	14 - 17 cm
35207 1013	Left	Large	17 - 20 cm
35207 1014	Left	X-Large	20 - 23 cm
35207 2010	Right	X-Small	< 11 cm
35207 2011	Right	Small	11 - 14 cm
35207 2012	Right	Medium	14 - 17 cm
35207 2013	Right	Large	17 - 20 cm
35207 2014	Right	X-Large	20 - 23 cm