# SELECTION<sup>®</sup>

**35206  
RIGID**

**35204  
35205  
SOFT**

SELECTION

SELECTION

## 35204, 35205, 35206 SELECTION WRIST SUPPORT SOFT & RIGID

Carefully read this instruction sheet before use!

### Intended use/Indications

#### Selection Wrist Soft

These products are intended for when the wrist needs support and off-loading in cases such as rheumatic disorders, soft tissue injuries or with degenerative changes.

#### Selection Wrist Rigid

These products are intended for wrists in need of support and off-loading of the first CMC-joint, in cases such as rheumatic disorders, soft tissue injuries or with degenerative changes.

### Indications:

Inflammations, work related strain injuries or when rheumatic and/or degenerative changes are present.

### Contraindications

No known.

### Adjustments

These products should only be altered by a medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

### Material Specifications


This product is made from polyester, nylon and elastan. Straps are made from plastic. This product is not flame retardant.

### Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

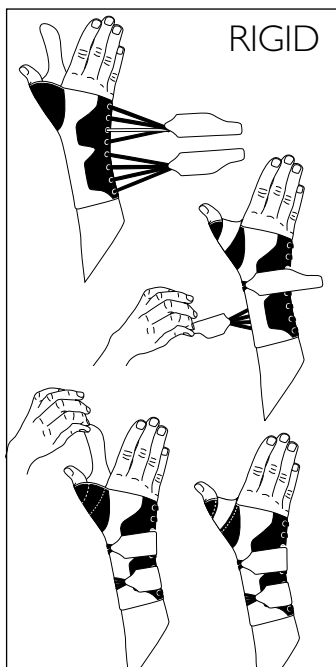## Instructions for use

1. Check that the support is complete and without damage.
2. Open the Velcro straps and release the lacing.
3. Pull the orthosis on like a glove.
4. Secure the two straps around the wrist.
5. Secure the thumb strap.
6. Adjust for comfort.

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