

# SELECTION<sup>®</sup>



**35206  
RIGID**

**35204  
35205  
SOFT**

SELECTION

SELECTION

## 35204, 35205, 35206 SELECTION WRIST SUPPORT SOFT & RIGID

Carefully read this instruction sheet before use!

### Intended use/Indications

#### Selection Wrist Soft

These products are intended for when the wrist needs support and off-loading in cases such as rheumatic disorders, soft tissue injuries or with degenerative changes.

#### Selection Wrist Rigid

These products are intended for wrists in need of support and off-loading of the first CMC-joint, in cases such as rheumatic disorders, soft tissue injuries or with degenerative changes.

### Indications:

Inflammations, work related strain injuries or when rheumatic and/or degenerative changes are present.

### Contraindications

No known.

### Adjustments

These products should only be altered by a medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your fit on a daily basis.

### Material Specifications


This product is made from polyester, nylon and elastan. Straps are made from plastic. This product is not flame retardant.

### Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

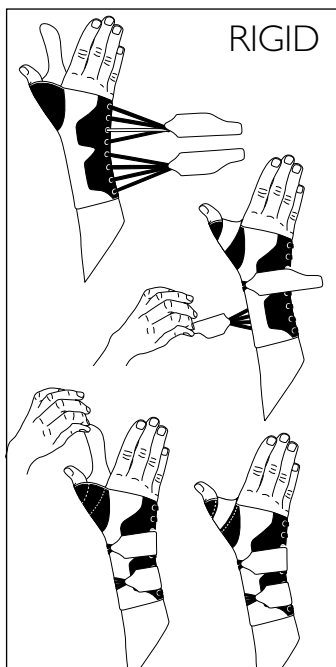

## Instructions for use

1. Check that the support is complete and without damage.
2. Open the Velcro straps and release the lacing.
3. Pull the orthosis on like a glove.
4. Secure the two straps around the wrist.
5. Secure the thumb strap.
6. Adjust for comfort.

Item No. SOFT Black	Item No. SOFT Beige	Item No. Rigid Black	Side	Size
35204 1010	35205 1010	35206 1010	Left	X-Small
35204 1011	35205 1011	35206 1011	Left	Small
35204 1115	35205 1115	N/A	Left	Small Plus
35204 1012	35205 1012	35206 1012	Left	Medium
35204 1125	35205 1125	N/A	Left	Medium Plus
35204 1013	35205 1013	35206 1013	Left	Large
35204 1135	35205 1135	N/A	Left	Large Plus
35204 1014	35205 1014	35206 1014	Left	X-Large
35204 1145	35205 1145	N/A	Left	X-Large Plus
35204 1015	35205 1015	35206 1015	Left	XX-Large
35204 1155	35205 1155	N/A	Left	XX-Large Plus
35204 2010	35205 2010	35206 2010	Right	X-Small
35204 2011	35205 2011	35206 2011	Right	Small
35204 2115	35205 2115	N/A	Right	Small Plus
35204 2012	35205 2012	35206 2012	Right	Medium
35204 2125	35205 2125	N/A	Right	Medium Plus
35204 2013	35205 2013	35206 2013	Right	Large
35204 2135	35205 2135	N/A	Right	Large Plus
35204 2014	35205 2014	35206 2014	Right	X-Large
35204 2145	35205 2145	N/A	Right	X-Large Plus
35204 2015	35205 2015	35206 2015	Right	XX-Large
35204 2155	35205 2155	N/A	Right	XX-Large Plus

Size	Wrist circumference	Arm circumference
X-Small	< 11 cm	N/A
Small	11 - 14 cm	N/A
Small Plus	11 - 14 cm	18 - 21 cm
Medium	14 - 17 cm	N/A
Medium Plus	14 - 17 cm	21 - 24 cm
Large	17 - 20 cm	N/A
Large Plus	17 - 20 cm	24 - 27 cm
X-Large	20 - 23 cm	N/A
X-Large Plus	20 - 23 cm	27 - 30 cm
XX-Large	23 - 26 cm	N/A
XX-Large Plus	23 - 26 cm	30 - 33 cm

