35306 SELECTION OPEN WRIST CHILDREN



ENGLISH

Intended use

To immobilize CMC and MCP I joint and wrist in order to reduce pain, to guide the wrist and an adducted thumb into a neutral position and to support a wrist after trauma/post operatively if needed. Intended for one person use.

Indications

CP, Rheumatoid arthritis fi thumb is adducted, post trauma.

Contraiandications

No known.

Adjustments

Fitter should make sure the stay is properly adjusted. This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

35306 SELECTION OPEN WRIST CHILDREN

Carefully read this instruction sheet before use!

Material Specifications

This product is made from polyester, Lycra and Nylon. Spoons are made from aluminum. This product is not flame retardant.

Washing Instruction


Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Remove stay before laundering. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.



Instructions for use

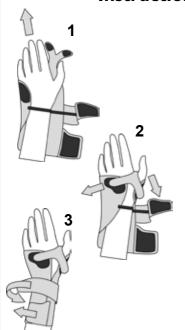

- Check that the support is complete and without damage.
- Bend the stay to fit the shape of the wrist (preferably with the stay in the pocket).
- Position the hand in the open brace (Image 1).
- Close the strap around the thumb and the strap on the dorsal side of the hand (Image 2).
- Close the circular strap and the strap on the arm (Image 3).
- 6. Adjust the straps around thumb into correct position.
- 7. Adjust all straps for comfort.





Item No.	Side	Size	Wrist circumference	Palm height
35306 1011	Left	Small	11,5 - 14 cm	7 - 8 cm
35306 1012	Left	Medium	12 - 15,5 cm	8 - 9 cm
35306 1013	Left	Large	14 - 16,5 cm	8 - 9 cm
35306 2011	Right	Small	11,5 - 14 cm	7 - 8 cm
35306 2012	Right	Medium	12 - 15,5 cm	8 - 9 cm
35306 2013	Right	Large	14 - 16,5 cm	8 - 9 cm