

# VixOne™ for DeVilbiss

Disposable Nebulizer (also serves as backup nebulizer when used in conjunction with VixOne for DeVilbiss Reusable Neb)

Manufactured by:



Somerset, PA 15501-2125 USA  
www.devilbisshealthcare.com  
800.338.1988



Rx Only



Latex Free



016958940343

ORDER #3655D-621  
UNITS/CASE - 50

**CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician. Single patient use. Not intended for reprocessing.**



## Instructions for Use

Includes Nebulizer, Tee, Corrugated Tube, 7" Patient Tube

1. This product is intended for use in delivering aerosol medications.
2. Remove nebulizer from the package and add prescribed medication in opening at the top of nebulizer.
3. Attach corrugated tubing to tee, then attach tee to nebulizer top. Attach mouthpiece to open end of tee.
4. Attach 7" tubing to bottom of nebulizer cup; connect other end of tubing to your DeVilbiss pulmonary drug delivery device's air outlet port.
5. Ensure all connections are secure; turn on your DeVilbiss unit. Confirm appropriate aerosol function. If necessary, tap nebulizer cup until mist appears.
6. Begin treatment as instructed by your physician. Continue until all medication has been aerosolized. Some of the medicine may cling to the sides of nebulizer cup during treatment. Occasionally tap cup to nebulize medication.



This product is designed for use with compressors; not tested for use with anesthetic or ventilator systems. Patients using oxygen therapy should never do so near ignition sources.

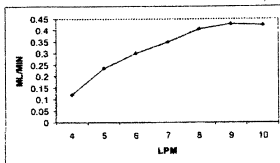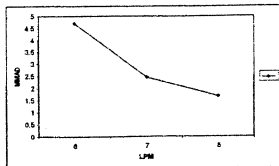

Performance information may not apply to drugs supplied in suspension or high viscosity form, consult drug supplier.

**LOT N14291**

**Westmed**

5580 S. Nogales Highway  
Tucson, Arizona 85706 USA  
Phone: 800-975-7987  
Fax: 520-294-6061  
www.westmedinc.com



MT Promedt Consulting GmbH  
Altenhofstr. 80  
66386 St. Ingbert, Germany  
ASSEMBLED IN MEXICO  
LABEL PN 71782, Rev. 01