

vive®

TOILET SEAT CUSHION

Owner's Manual - CSH1061WHT



HOW TO ATTACH TO TOILET SEAT

1. Make sure the toilet seat and suction cups on the Toilet Seat Cushion are clean.



2. Line up the cushion with the toilet seat and ensure there is enough clearance in the toilet seat opening.



3. Place the cushion on the toilet seat

4. Press down lightly to make sure the suction cups attach securely.



HOW TO REMOVE FROM TOILET SEAT

Option 1: Lift the Toilet Seat Cushion gently and pull upward on the suction cup pull tabs.



Option 2: Lift the Toilet Seat Cushion gently and slide the putty knife under the suction cups to remove.



HOW TO REMOVE FOAM FROM COVER

- Locate the zipper on the bottom of the Toilet Seat Cushion.
- Unzip completely and remove the foam from the cover.



CARE INSTRUCTION

- Wipe clean with a disinfecting wipe or soap and water.
- Dry completely.