




## Easy Wipe Gait Belt

Single Patient Use Only • Rx Only

**Product Number**  
M5168

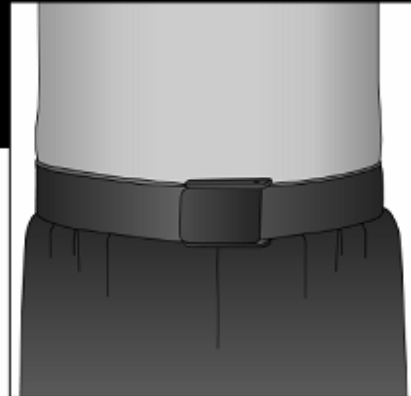

### IMPORTANT INFORMATION

Please read all instructions before use. Correct application is essential to proper functioning of product.

### INDICATIONS

Patients that require assistance with ambulation.

### PRECAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which they are responsible. The physician or practitioner is responsible for providing use instructions and precautions to other healthcare practitioners, care providers involved in the patients' care and the patient.
- Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or prolonged pain while using this product.
- Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

### INSTRUCTIONS FOR USE

1. Wrap the gait belt around the patient.
2. Open the lever on the buckle and slide the strap through the buckle slot.
3. Pull the strap through the buckle to tighten.
4. The buckle should be adjusted to a snug, but comfortable fit for the patient. Verify the buckle is properly closed.
5. Check for adequate circulation by placing two fingers width between the belt and the patient.



### WASHING INSTRUCTIONS

Wipe clean with mild disinfectant. Do not submerge product in a cleaning solution for an extended period of time.