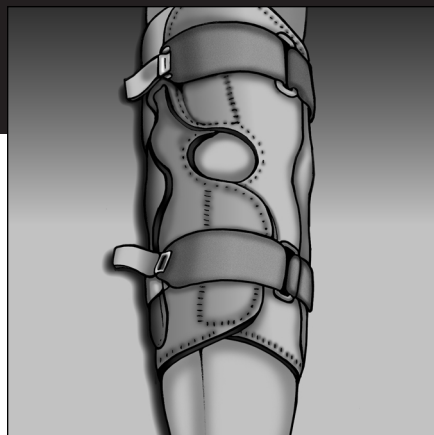


Wrap Around Hinged Neoprene Knee Support

REF 0-844



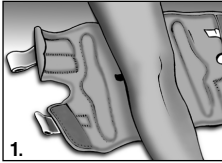
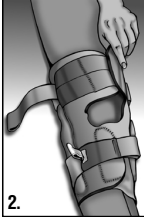
Intended Use

This neoprene wrap around knee support is for patients with various types of knee injuries including, but not limited to, mild knee instabilities and osteoarthritis. It is designed to provide warmth and compression along with medial and lateral stability to the knee. The wrap around design facilitates application and removal.

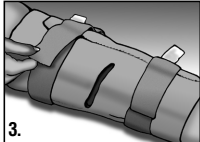
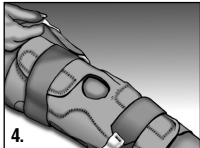
Precautions

Consult your medical professional immediately if you experience sensation changes, unusual reactions, swelling, or prolonged pain while using this product.

Directions For Use

1. Open support and place underneath the patient's extended (straightened) leg. Inside tag should be proximal or closest to the hip.
 
2. Place the hinges along the side of the leg so that the hinge axis aligns with the center of the knee. Secure the support around the anterior (front) leg, fastening the hook and loop closure on the neoprene.
 
3. After ensuring that the hinge is aligned properly, secure the hook and loop closure straps on the posterior (back) aspect of the splint

to maintain the hinge alignment. (Straps may be trimmed if needed.)

4. At this time, re-secure the support around the anterior (front) leg fastening the hook and loop closure on the neoprene to provide a snug, but comfortable fit.
 
5. Once the support is in place, secure the superior (top) and inferior (bottom) hook and loop straps.
 

Knee Circumference

X-Small	10.0" – 11.0"	Large	15.5" – 18.0"
Small	11.0" – 13.5"	X-Large	18.0" – 22.0"
Medium	13.5" – 15.5"	XX-Large	22.0" – 24.0"

Caution: Federal U.S.A law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner.)



Manufacturer

DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828

Authorized Representative

DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.80100

